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## 4- Wheel Aluminum Rollator

Wersja/Version: MOBILUS LITE

Typ/Type: VCBK

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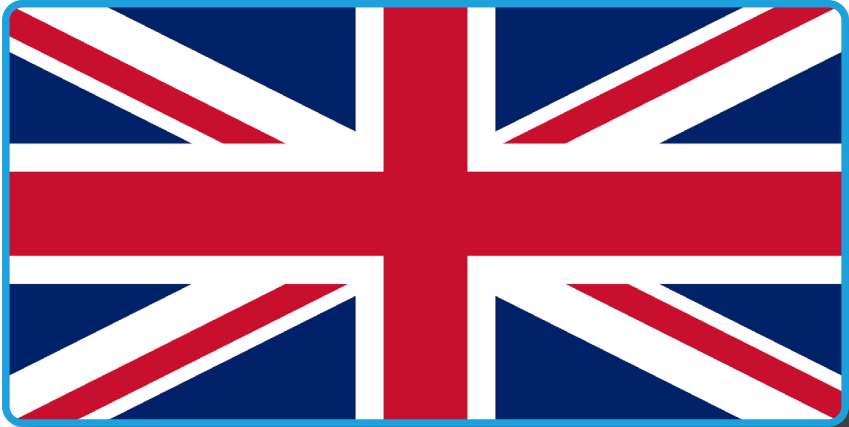
**English version** on next page

# Table of contents

Indications.....	18
Contraindications .....	18
<b>Product description.....</b>	<b>18</b>
List of elements in the package.....	19
<b>Usage .....</b>	<b>19</b>
Mounting and dismounting the rollator.....	19
Height adjustment of handles .....	20
Braking system .....	21
Folding.....	21
Overcoming obstacles (thresholds, kerbs).....	22
<b>Safety instructions .....</b>	<b>22</b>
<b>Technical specifications .....</b>	<b>24</b>
<b>Service and repair.....</b>	<b>24</b>
<b>Care and maintenance .....</b>	<b>25</b>
Cleaning.....	25
Disinfection.....	25
Product disposal .....	25
Maintenance plan.....	26
<b>Symbols .....</b>	<b>26</b>
<b>Contact details.....</b>	<b>28</b>

## Indications

Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

## Contraindications

Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

## Product description

Illustration 1.

## List of elements in the package

Illustration 2. Items that should be included in the package.

## Usage

### Mounting and dismounting the rollator

#### WARNING!

Special care must be taken when folding the rollator as there is a risk of jamming fingers or hands.

- Product packaged for assembly.
- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the manufacturer.
- Do not use the rollator until all parts are properly connected.
- Unfolding rollator should begin unfolded anchor hinge by pressing both hand seat frame till the moment when You can hear a click. (Illustration 3.)
- Then unscrew the screws attaching the shoulder of the rear wheels. Lower the rear wheels so that you can tighten the screws in the holes of the arms of wheels. Make sure that the screws are correct screwed in.

- Then unscrew the screws attaching the shoulder of the front wheels. Lower the front wheels so that you can tighten the screws in the holes of the arms of wheels. Make sure that the screws are correct screwed in.

Illustration 3. Unfolding the frame.

Illustration 4. Wheels down

- Bag for small items should be attached with velcro to the rollator side frames. (Illustration 5.)

Illustration 5. Bag attached.

## Height adjustment of handles

- Handles should be located at a height that will provide comfort and a sense of security for the user. It is important to ensure that both hands are positioned at the same height.
- To change the height of the handles, press the pin located on the frame (Illustration 6.) and, depending on the needs extend or retract the frame handles.
- After adjusting the correct height, the pin should appear in one of the holes of regulation. At this point should be audible click.

Illustration 6. Height adjustment of handles.

## Braking system

- The support is equipped with a multi-functional brake. The brake handle is located under the handle. It can operate as a standard brake and as a parking brake.
- To use the brake as standard, pull the brake handle upwards.
- To use the parking brake, push the brake handle down until you can hear the characteristic sound of the brake being applied (use the right and left brake at the same time). This will cause the rear wheels to lock.
- To unlock the wheels, pull the brake handle upwards.

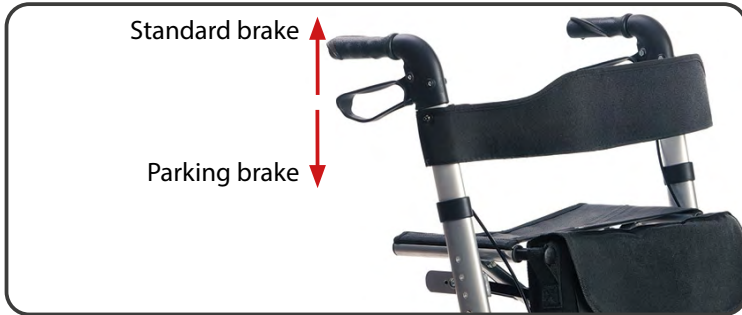
Illustration 7. Brakes operation.

## Folding

- The rollator can be folded for ease of transport.
- Pull the handle located in the center of the seat (Illustration 8.), which will fold the cross frame of the support.
- In addition, you can reduce the size of the rollator by folding the wheel arms. To do this, it is necessary to unscrew the wheel arm mounting screws, raise the arms to the position shown in Illustration 9.
- Screw in the bolts bypassing the holes of the wheel arms. Make sure that the screws are properly screwed in.

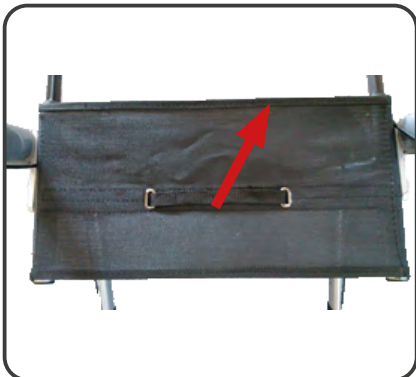
Illustration 8. Seat folding

Illustration 9. Rollator folded

## Overcoming obstacles (thresholds, kerbs)

- In order to drive the rollator onto the threshold or curb should set the rollator perpendicularly to the edge of the obstacle. Holding the handle, tilt the rollator in rear and lightly keep by the foot one of the footer located at the rear wheel, which will raise the front wheels up. Drive the rear wheels to the edge of the obstacle and holding the rollator handle, lift slightly up and enter rollator on the obstacle.

## Safety instructions

### WARNING!

Any 'serious incident' relating to a device must be reported to the manufacturer and to the competent authority of the Member State in which the user is resident.

### ATTENTION!

The product is intended for people with disabilities and musculoskeletal conditions. If you are unsure whether your health condition allows you to use the product, consult a healthcare professional.

### WARNING!

The product must not be modified in any way; this will void the warranty.

### WARNING!

Exceeding the user's maximum weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

### WARNING!

Please read these instructions for use before using the rollator and keep them for future reference.

- Before using the rollator, refer to this manual and keep it for future reference.
- At standstill and the use of seat should apply the locking rotation of the wheels both rear wheels.
- The stand will not replace the wheelchair, the seat is used only for temporary resting or transport the patient on a short distance.
- The rollator is used only to provide rest to one user. Do not stand on it.
- The rollator should be used on even surfaces at an incline less than 5% (2,86°).
- With the rollator you can only use on surfaces where all four wheels touching the ground are to use all of four wheels.
- Use of the product on the icy, wet, greasy, strongly gritty surfaces is unacceptable - it creates a risk of falling.

- Before each use of this product:
  - Perform visual inspection of all structural components of the product,
  - Check the operation of the braking system. Squeeze the brake handles to see if the brake pads are fit properly to wheels and are able to stop rollator. Press the handles down to activate the parking brake and check if it works properly.
  - Carefully check the stability and condition of the support frame.
- Do not use the support in the event of any visible or perceptible damage (e.g. cracks in the frame). It may cause an accident.
- Maximum weight that can be moved in the bag - 1,5 kg.
- Do not leave the stroller with the user close to open sources of fire.
- Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of stroller. It may cause burns or frostbite parts of the body.
- All repairs must be carried out by qualified personnel.
- And during the warranty period only by the manufacturer's service.
- Do not exceed the maximum height of the handles to push, rollator will lose stability.
- Push the rollator with both hands simultaneously in the direction of travel.
- Before using, make sure that all screws are tight.
- Make sure that the Levels hinge of stretch securing before submitting rollator, work properly.
- Due to the possibility of jamming the fingers put extreme caution in the area of horizontal hinge strut, fold-out footrest and side frame hinge.

**WARNING!**  
Maximum user weight is 136 kg

## Technical specifications

Maximum user weight	136 kg
Maximum support height	795-920 mm
Height adjustment steps	6
Adjustment range	25 mm
Maximum bag load	1.5 kg
Support width	660 mm
Distance between handles	495 mm
Seat height	530 mm
Seat width x length	455 x 225 mm
Rollator length	685 mm
Folded rollator dimensions	660 x 330 x 290 mm
Rollator Weight	7,47 kg
Minimum turning radius	95°
Front wheel diameter	193 mm
Rear wheel diameter	170 mm
Wheels type	Peñe
Maximum safe slope	5% (2,86°)
Materials	Aluminium, polietylen
Description of the environment in which the product will be used	  
Storage and usage temperature	From +5°C to +40°C
Air humidity for storage and use	From 30% to 70 %

## Service and repair

The inspection should be carried out regularly and should be preceded by a thorough cleaning of the product (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, please contact your distributor.

# Care and maintenance

## Cleaning

- Clean the support frame with a damp cloth, the seat with mild detergents.
- Remove dirt from moving rollator components (wheels, bolts, clamp locks) on an ongoing basis.
- In order to maintain the performance of the product, it is extremely important to keep all its components clean and store in appropriate conditions. Storage in rooms with very high humidity (baths, laundries, etc.), or in the open air, combined with a lack of hygiene, will result in a rapid loss of performance and aesthetics.
- Do not expose the rollator to extreme weather conditions (rain, snow very strong sun), as certain components may change colour.

### **Must not be used:**

- solvents
- toilet cleaners
- sharp brushes and hard objects
- chlorine-containing agents
- corrosive cleaning agents

*Do not clean the rollator with pressure or steam unit!*

## Disinfection

If the product is used by a large number of people (e.g. in a nursing home), commercially available disinfectants should be used. Prior to disinfection, the top surface should be cleaned. Only tested and approved disinfectants may be used for disinfection. Information on recognised and approved disinfectants and disinfection methods can be obtained from the national health authority.

### **WARNING!**

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

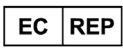
## Product disposal


The product must be disposed of in accordance with current local and national regulations. For proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, contact your local waste disposal company or authorised distributor.

## Maintenance plan

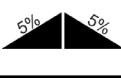Frequency	Subject of the inspection	Comments
Before the first installation	Checking the condition of all components in the package	Check that no components have visible mechanical damage
Before each use	Wheel and brake inspection	Brakes can wear out, check the condition of the brakes before each use. Wheels should be checked for wear and cleaned of dust and sand
Every two weeks	Screws and nuts - making sure they are securely fastened	Ensure that all screws and nuts are tightened securely
Every two months	Cleanliness, general condition	In the event of mechanical damage, contact the manufacturer immediately
Once a month	Wheel axles	Wheel axles must be maintained, by spraying a few drops of preservative-lubricant between the axle and the hub

## Symbols

	Medical device
	Production date (printed on the product)
	Manufacturer
	Distributor
	Authorised representative in the EU
	Conformité Européene (conformity with EU requirements)
	Catalogue number (to be reordered)

	Serial number
	LOT number
	Unique identification code
	Refer to the instructions for use
	Warning
	Possibility of jammed fingers
	Permitted temperature
	Protect from moisture
	For indoor/outdoor use
	Keep away from sunlight
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Maximum user weight

## Contact details

### Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

[www.mdh.pl](http://www.mdh.pl) [www.viteacare.com](http://www.viteacare.com)

We reserve the right to make technical changes due to technical progress.