



SAFE LIVING AT HOME

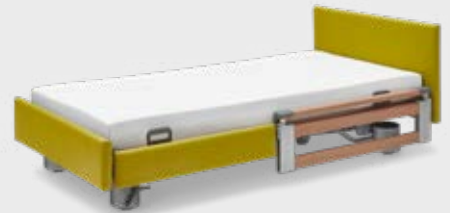

Retailer catalogue

Beds and accessories for caring at home

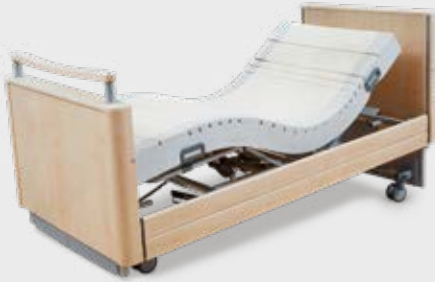

International edition

STIEGELMEYERGROUP



















Contents


Good to know

- 3 Burmeier, the company
- 4 Our support for your processes
- 6 Safety standards for care beds
- 7 The 24-volt drive system
- 8 Econ drive system
- 10 Easy Switch for our Regia bed
- 13 On a safe trajectory through the challenges of care at home
- 14 Technical data
- 21 Dali range
- 39 Lippe range, Relax
- 57 Regia, Inovia
- 77 Westfalia range
- 87 Lenus
- 93 Heavy duty beds
- 103 Bedside cabinets and side tables
- 107 Mattresses, transport chair and furniture
- 119 The Stiegmeyer-Group

Care bed and bed inserts

22 Dali	26 Dali wash	30 Dali low-econ	34 Dali low-entry
			
The top-selling care bed in Germany	The machine-washable care bed	The practical low-height bed	The practical low-height bed
40 Lippe IV	44 Lippe IV washable	48 Lippe 120/140	52 Relax
			
The flexible bed insert	The bed insert for automated reprocessing	The comfortable bed insert	The combinable senior bed
58 Regia	66 Regia partner	72 Inovia	
			
The versatile care bed for the discerning	The flexible senior bed for couples	The innovative low-height bed	
78 Westfalia IV	82 Westfalia-Klassik	88 Lenus	
			
The feelgood care bed	The comfortable care bed	The low-height bed for optimal fall prevention	
94 Gigant	98 Allura II		
			
The heavy-duty bed that meets the highest standards	The heavy-duty bed with superior comfort		

Bedside cabinets

104

Practical companions for every bed

Mattresses, transport chair and furniture

108	110	114
		
Well-being through lying comfort	Ravello – the care chair for extra mobility	Home furnishings for every requirement



Burmeier – safe living at home

In the near future, caring for relatives at home will influence social life in Europe like hardly any other topic. Modern care beds make a significant contribution to helping different generations live together successfully. Burmeier has been the leading specialist for home care beds and furniture in Germany for decades. From our new location in Hiddenhausen, Westphalia, we can meet your requirements even faster and more conveniently.

Take advantage of these benefits:

- An ultra-modern digital logistics centre makes nationwide delivery times of just 24 to 48 hours possible
- More than 120 years of experience with medical beds, as part of the Stiegmeyer-Group
- Family-run company with innovative spirit
- Promoting contemporary care through modern concepts such as low-height beds and split safety sides
- Homelike beds with many design options for private customers
- The highest safety and quality standards, assured by extensive product testing above and beyond the standard requirements

At Burmeier, we attach great value to sustainability

Ethical behaviour, environmental protection and the elimination of discrimination are essential goals for every company today. As part of the Stiegmeyer-Group, Burmeier has been actively involved in shaping this development for many years. Respect, tolerance, honesty and fairness towards each other and towards our customers are our guiding principles. We are also pursuing many ambitious goals in the area of sustainability.

Careful use of raw materials:

In our wood production facility in Nordhausen, 95% of the wood and wooden materials used come from sustainable forestry and are certified according to FSC or PEFC. This means that for every tree felled, a new tree is planted.


Environmental protection and energy efficiency:

With solar systems on its roofs, the Stiegmeyer-Group feeds energy into the power grid and helps to reduce CO2 emissions. Our company operates a certified energy management system in accordance with DIN EN ISO 50001. Our state-of-the-art fleet of lorries and carefully optimised route planning also help ensure low CO2 emissions.

Digitalisation:

In our everyday working lives, digitalisation helps to conserve resources. Paper is becoming rare in our offices. Instead of travelling long distances to meetings, we hold many meetings as video conferences. In our international company, with locations from Finland to South Africa, this results in a significant reduction in emissions.

Our support for your processes

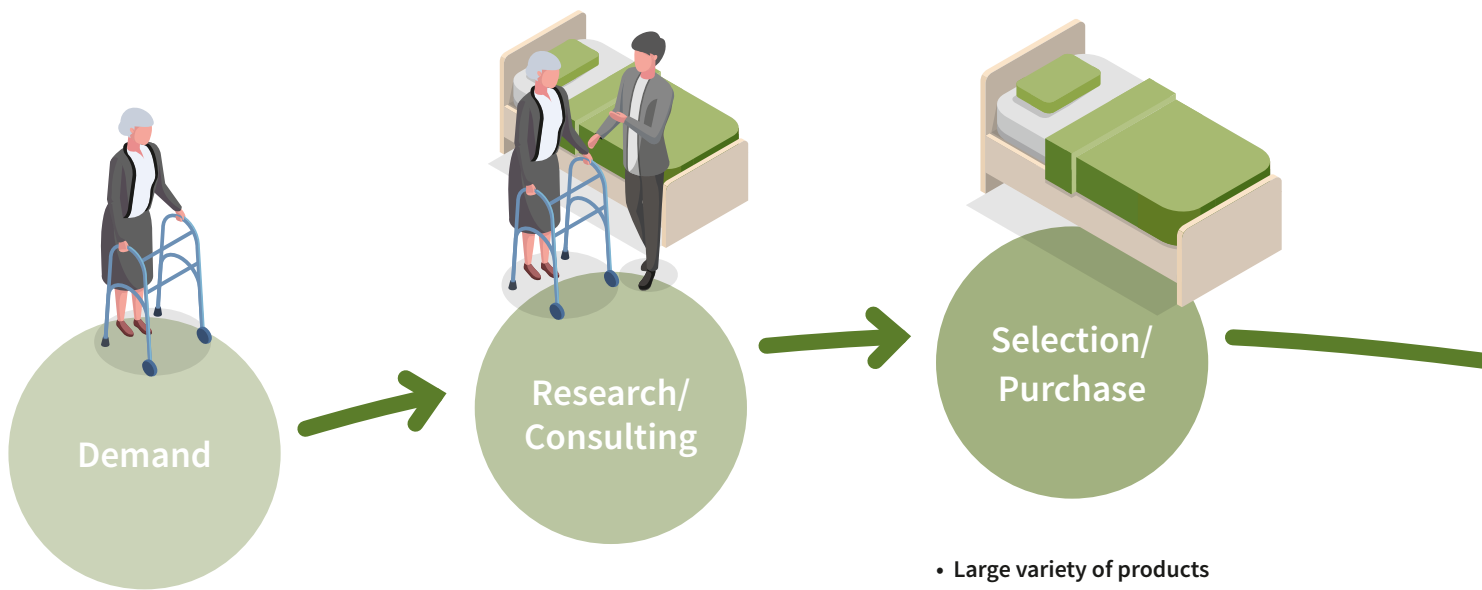

In many countries, the demand for age-related and medically-indicated aids such as care beds is driven by demographic change. In Germany alone, for example, more than 3.3 million people are cared for at home.*

*(Source: Statista)

- Customer-centred product descriptions
- Dealer search under burmeier.com
- Easily understandable, comprehensive information materials
- Optimal advice via configuration sheets
- The right care bed to suit specific medical requirements

- Large variety of products
- Telephone support
- Personal consultation and assistance from our retailers
- The highest quality and safety standards
- Reliable spare parts service
- Extensive cross-selling and up-selling potential thanks to bedside cabinets and mattresses




Robert Spencer,
Branch Manager,
medical supplies store

“Burmeier supplies me with high-quality products and spare parts with long-term availability. The company’s competent service is always available and gives me the security I need for my retail business with care beds.”




Maggy Smith,
occupational therapist

“A high level of comfort and safety combined with benefits regarding care. Burmeier’s care beds offer ideal support during everyday work.”

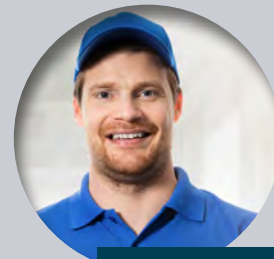

Dave Johnson,
rehabilitation clinic
technician

“Burmeier beds can be assembled effortlessly and really quickly. The high quality of the beds keeps the claims quota low – and if we need replacement parts, they are delivered promptly. The personal support provided by Burmeier is exemplary.”

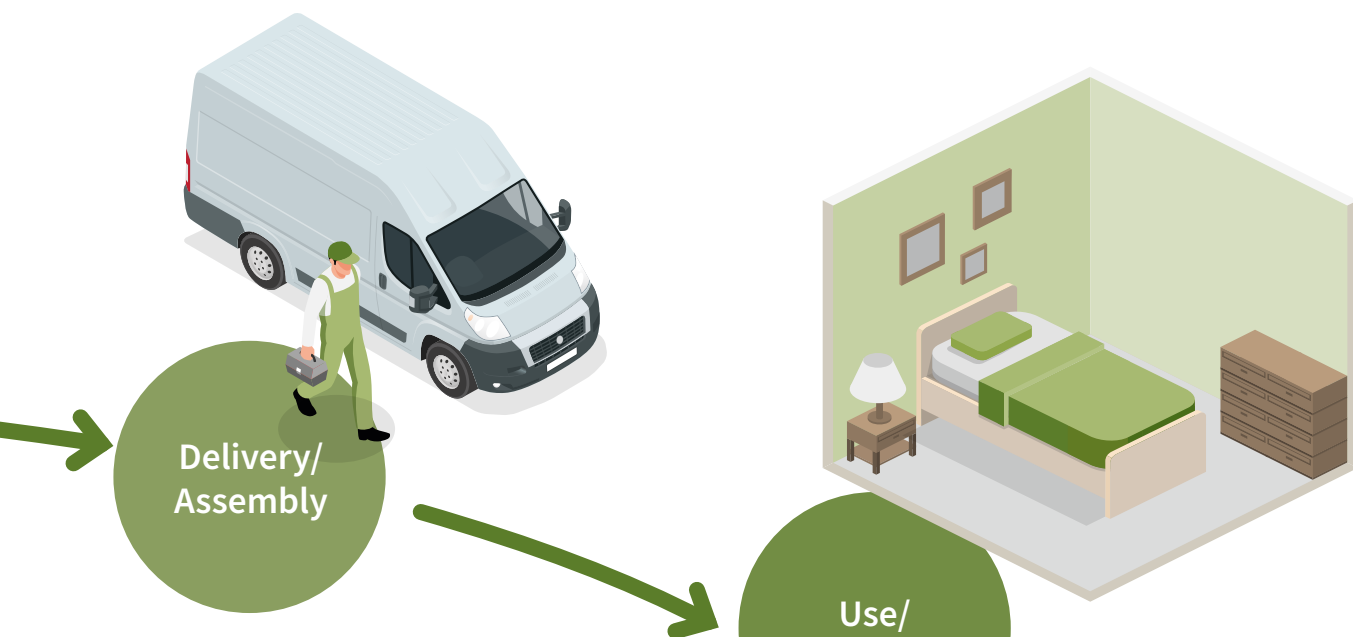

- Modern digital logistics centre
- Lean warehousing and uncomplicated replacement thanks to common parts strategy
- Quick and reliable delivery
- Space-saving installation of all components
- Easy one-person assembly
- Helpful assembly videos

- Optimal user-friendliness, functionality and durability
- The high quality of the products guarantees a low claims and repairs quota

Recommendations

- The satisfaction of all customer segments results in a high number of recommendations



Cathleen Turner,
customer

“When I was suddenly faced with finding a care bed, I opted for Burmeier. The homelike design, comfort and high level of safety were what convinced me straight away. It’s great to find so much quality at such a good price/performance ratio.”

All information at your fingertips

In the dealer section of our website, you always have access to:

- Replacement parts lists
- Instruction manuals
- Certificates
- Product photos
- ...and much more – around the clock!

Register now at
partner.burmeier.com/en/



Our key focuses and innovations



Safety standards

All beds are designed for maximum safety as standard

This includes:

- Primary fuse, electronic overload cut-off device and thermal fuse against overheating
- Motors run on safety extra-low voltage
- In-company quality standards exceed industry standard requirements
- Anti-kink strain relief on bed frame and on motor housing
- At least IPX4 splash-proof
- Optional IPX6 splash protection for automated reprocessing

The following standards and directives are applied

- Medical Device Regulation MDR – (2017/745)
- DIN EN 62353: Medical electrical equipment – Recurrent test and test after repair of medical electrical equipment
- DIN EN 12182: Technical aids for disabled persons
- DIN 32977: Adapted designing for disabled persons
- DIN EN 60601-1: Medical electrical equipment – General requirements for basic safety and essential performance
- DIN EN 60601-1-2: Medical electrical equipment – Collateral standard: Electromagnetic compatibility
- DIN EN 60601-1-6: Medical electrical equipment – Collateral standard: Suitability for intended use
- DIN EN 60601-2-52: Medical electrical equipment – Particular requirements for basic safety and essential performance of medical beds
- DIN EN ISO 14971: Medical devices – Application of risk management to medical devices



The SwitchMode power supply unit produces a 24-volt safety extra-low voltage directly at the socket.

24-volt drive system

Burmeier's innovative 24-volt drive system provides an ideal safety standard for caring for people at home. All live 230-volt components are banished from the care bed. Immediately downstream of the mains plug is a housing containing the **SwitchMode** electronic voltage transformer. Residents therefore enjoy a high level of safety.

You have less work to do and your customer saves electricity!

The bed has no current-carrying components except when the handset is in use. Power consumption drops by up to 93% as a result. Even more costs and effort are saved by the fact that **leakage current measurements are no longer necessary**, as certified by **TÜV Rheinland**. In principle, the required leakage current measurement must be carried out every 2 years for care beds in commercial use. The use of the 24-volt drive system prolongs this interval so that measurements are not necessary within the expected lifetime of the care bed. The applicable lifetime is indicated in the operating instructions.

Even when taking the bed into service, leakage current measurements are unnecessary as these have already been carried out in the factory. It is also not necessary for **an electrically trained person** to be involved in assembling the bed – this saves the bed operator additional costs.




Robert Spencer,
Branch Manager,
medical supplies store

“Safety and compliance with regulatory standards are important aspects when procuring care beds. Burmeier is the trusted partner that we rely on here.”

SAFE LIVING AT HOME


Energy saving

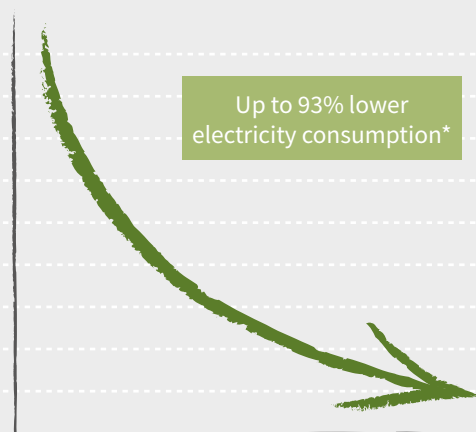

* Compared to 230-volt systems, assuming about 5 adjustments per day

SAFE LIVING AT HOME

Strong advantages

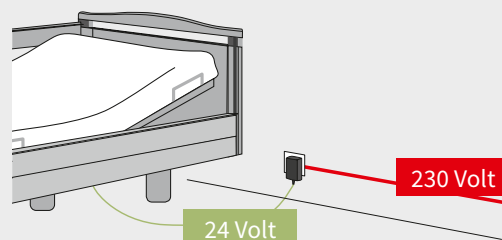

- Safety extra-low voltage starting from the transformer at the power socket
- No 230-volt components on the bed
- No leakage current measurements are necessary after installation or beyond during the expected service life of the bed
- A test certificate can be downloaded for every bed

Econ drive system



Most bed models in the Dali range are equipped with an electricity-saving 24-volt drive system as standard. In some countries, however, power plug standards apply that require the use of accessories in order to be compatible with the 24-volt drive system and its SwitchMode mains plug transformer. In this case, our Econ models with an **alternative drive** are an attractive, cost-effective choice.

Both the Dali and the Dali low-entry are available in the versions **Dali econ** and **Dali low-entry-econ**. A third model, the **Dali low-econ**, is also available, which does not exist in a 24-volt version.

All Econ beds have a safe, reliable and economical **230-volt drive**. With matching selectable mains plugs, these care beds are ready for international use wherever they are needed. The Econ model **handsets** are connected to the bed with a cable in the usual way. The handset can be used to selectively lock and unlock all adjustment functions. The Dali-Lock app cannot be used with the Econ beds.



The Dali low-econ budget-oriented low-height bed.



The Dali low-entry-econ low-height bed for ideal fall prevention.



The Econ drive system is available with many international power plugs.



The cabled handset of the Econ models can be attached flexibly to the safety side.

Easy Switch


Easy Switch for our Regia bed

The patented system for adaptable care beds with standard-compliant safety

Choose the Regia with the telescopic split safety side or without a safety side and benefit from our flexible Easy Switch system. The Easy Switch principle is as easy as it is impressive: The safety side elements, the headboard and footboard as well as the side panels can be attached and removed **without tools**. This is possible thanks to easy-to-use locks that snap into place securely and cannot be accidentally opened by the resident. Headboards and footboards from the Nobla range and also panels are available as an option with **attractive fabric or faux leather upholstery**.

Modular design for tool-free assembly

Thanks to the Easy Switch system, the Regia can easily be fitted any time with safety sides or a new look and **adapted to your requirements**. It becomes a bed that develops with you and supports you in every phase of your life. The removable safety sides are quick to convert to whatever care situation may arise. They also make it considerably easier to assemble the bed.

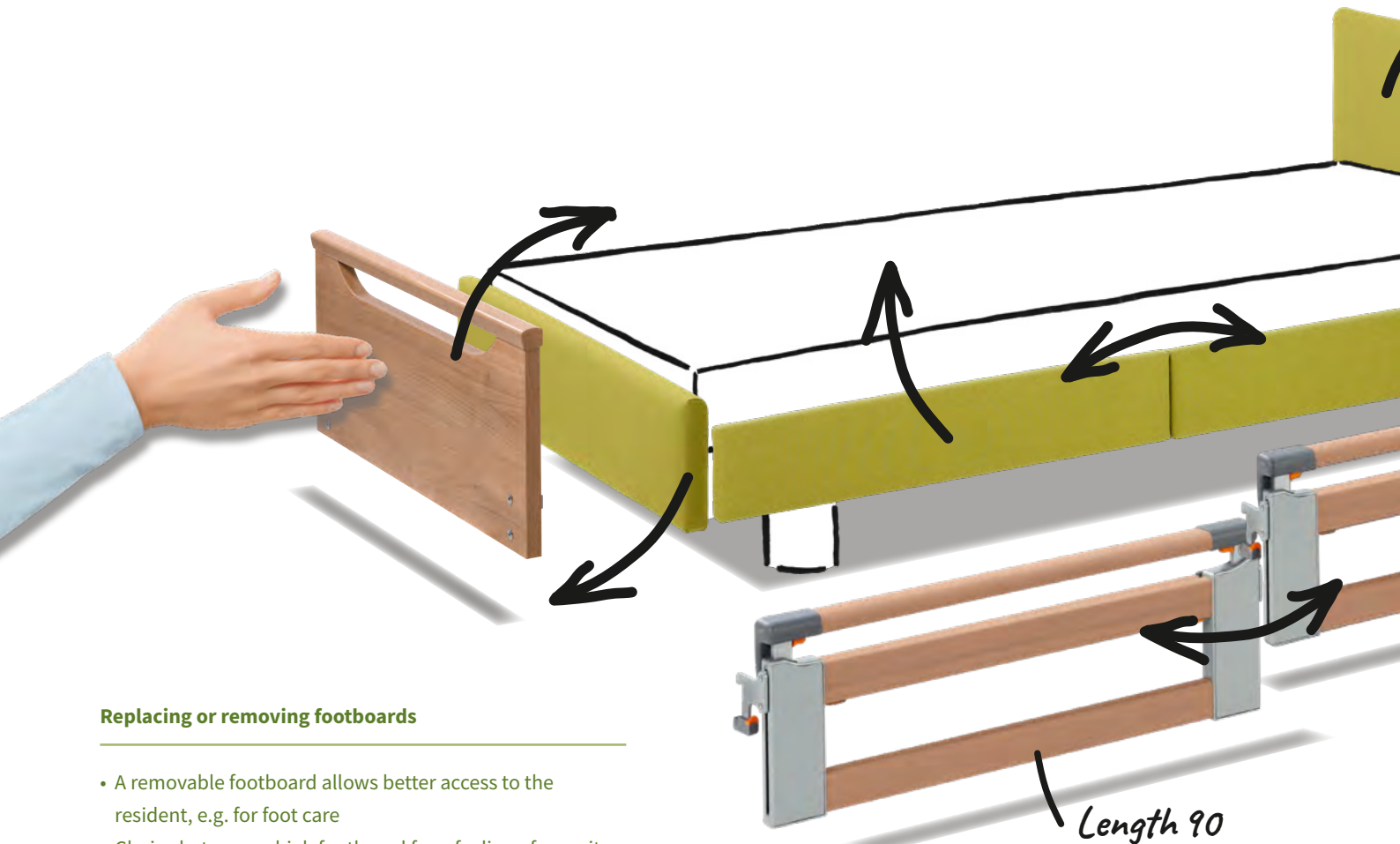

Replacing or removing footboards

- A removable footboard allows better access to the resident, e.g. for foot care
- Choice between a high footboard for a feeling of security or a low footboard for an unobstructed view
- Attractive wood decors, fabrics and faux leather available to choose from

**! Safety sides can be freely combined:
110/90 or 90/110**




James Borrow,
medical product consultant

“A bed that can be used and extended flexibly right from the start: for me, the perfect selling point.”

Replace headboards in seconds whenever required

- Homelike design variants, with optional handle bars
- Attractive wood decors, fabrics and faux leather to choose from
- Headboard, footboard and side panels freely combinable however you wish

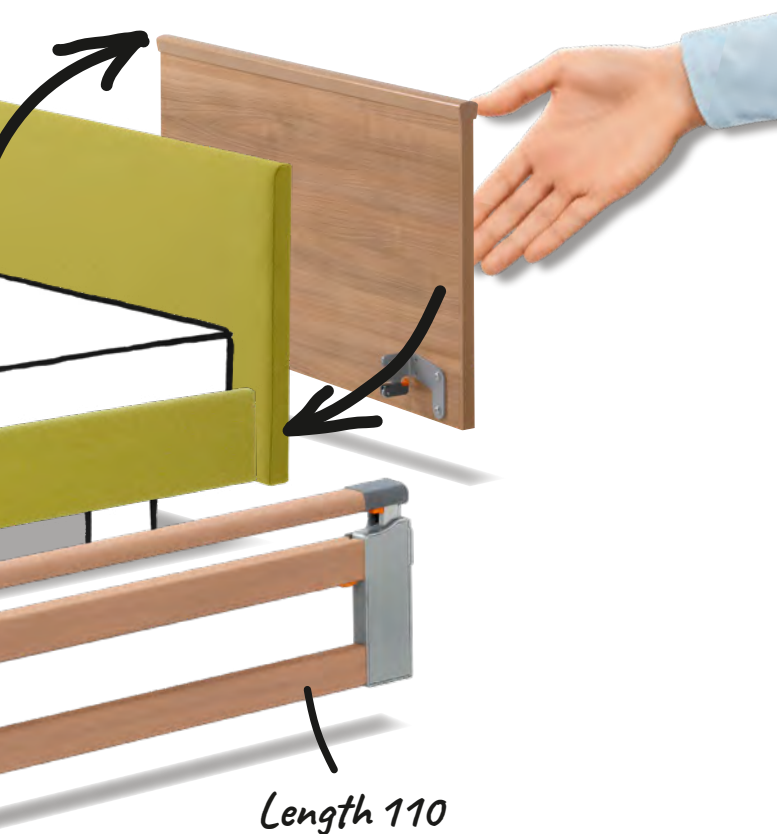

Attach or remove split safety sides as required

- Two elements with lengths of 90 and 110 cm can be freely combined or used separately
- Use of only the 110 cm safety side element at the head end provides standard-compliant protection
- Use of only the 90 cm safety side element at the head end provides reassurance and orientation and leaves ample space to get in and out of bed

SAFE LIVING AT HOME

Easy Switch for added flexibility and safety

Beds with Easy Switch meet the different protection needs of residents and can be customised to suit any taste. The flexible safety side provides standard-compliant protection without restricting the mobility of the person in need of care.



- ▲ All Easy Switch elements are inserted into the bed frame and attached with easy-to-operate locks.



- ▲ Simply press the fastener upwards.



- ▲ The lock snaps into place securely – and the element is attached.



- ▲ To release the fastener again, pull the orange safety catch towards you and press down on the lock.

On a safe trajectory through the challenges of care at home


The number of people in need of care is rising even faster than expected. According to a current Barmer care report, there will be six million people in need of care in Germany by 2030: one million more than previously estimated. At the same time, the nursing staff shortage is worsening. Home care, which already accounts for 80 per cent of care cases today, will continue to gain in importance in the coming years.

In this situation, technically innovative aids and smart solutions are our strongest allies. Care beds from Burmeier provide noticeable physical relief for caring relatives. Low-height beds and split safety sides promote the mobility and health of the user while helping to reduce physical labour.


The medical supply trade and Burmeier cooperate successfully to guarantee a secure supply of care beds, mattresses and bedside cabinets. From versatile standard beds to heavy-duty beds, the joint portfolio meets all requirements. The technical possibilities range from moderate support in everyday life to a low-entry bed for fall prevention in severe dementia cases. Models such as the Regia and the Inovia are also an attractive option for private customers and open up new business opportunities for specialist dealers.

Individual support is also provided for these tailor-made products. Our Area Sales Managers and Burmeier's Marketing Department are always available as experts for you to contact. First-class service and comprehensive advice go without saying in our family-run company.

Population above 65 years old in 2020^{*1}
in % of the entire population

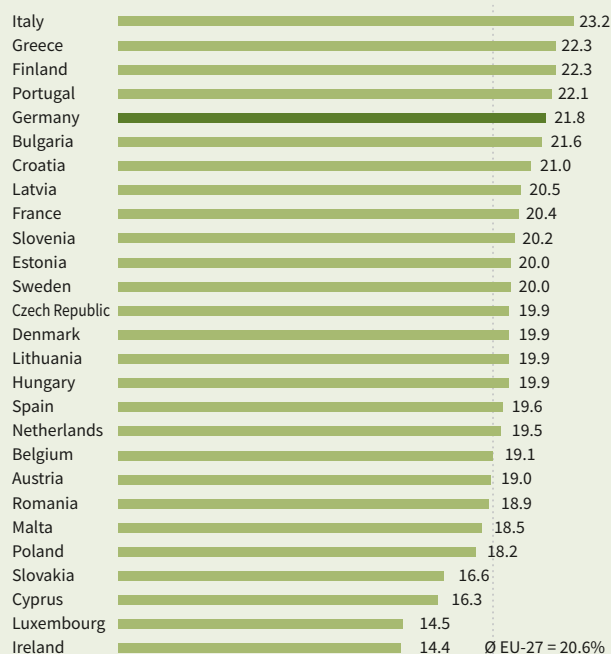

What would a new, customised care bed be worth to you?^{*2}

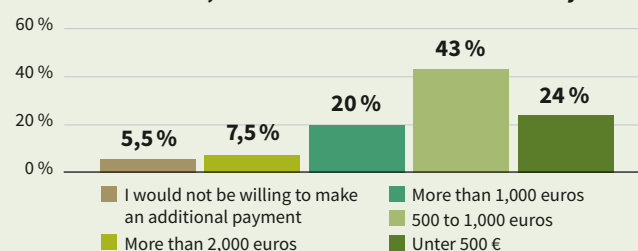

* Sources:

¹Eurostat © Statistisches Bundesamt (Destatis), 2021

²Appinio questionnaire, commissioned by Burmeier GmbH & Co. KG (06/2021)

Technical data

Model		Dali	Dali wash	Dali low-econ	Dali low-entry	Lippe IV	Lippe IV washable	Lippe 120/140
Adjustments								
Height	Electric	✓	✓	✓	✓	✓	✓	✓
Backrest	Electric	✓	✓	✓	✓	✓	✓	✓
Thigh rest	Electric	✓	✓	✓	✓	✓	✓	✓
Locking feature	Selective	✓	✓	✓	✓	✓	✓	✓
Locking feature	Central	–	–	–	–	–	–	–
Wireless handset		✓	✓	–	–	–	–	–
Cabled handset		O	–	✓	✓	✓	✓	✓
24-volt drive system		✓	✓	–	✓	✓	✓	✓
Econ drive system		O	–	✓	O	–	–	–
Height adjustment range								
from – to	cm	40-80	40-80	32-72	23-63	29-74	29-74	29-74
4-section mattress base								
Standard	cm	90 x 200	90 x 200	90 x 200	90 x 200	90 100 190 200	90 x 190 90 x 200	120 140 x 200
Special mattress-base widths		–	–	–	–	80	–	–
Metal slats		O	✓	–	✓	–	✓	✓
Wire mesh mattress base		–	–	–	–	–	–	–
Sprung wooden slats		✓	–	✓	–	✓	–	–
Comfort mattress base		–	–	–	–	–	–	–
One-piece		–	–	–	–	–	–	–
Multi-section		✓	✓	✓	✓	✓	✓	✓
Adjustable mattress base tilt		✓	✓	✓	✓	–	–	–
Backrest retraction		–	–	–	–	–	–	–
Ground clearance for lifting equipment		✓	✓	✓	✓	–	–	–
Castor locking								
Central		–	–	–	–	–	–	–
Per axle		–	–	–	–	–	–	–
Single		✓	✓	✓	✓	–	–	–
Castor diameter	mm	100	100	100	75	–	–	–
External dimensions and weight								
Length		218	218	225	225	185/195	185/195	195
Width		102	102	102	102	85/95	85	115/135
Net weight	kg	97	106	97	106	80	80	132 140
Safe working load								
	kg	185	185	185	175	200	200	250
Wooden surround								
Standard		–	–	–	–	–	–	–
Model-related headboard, closed surface		✓	✓	–	✓	–	–	–
Model-related footboard, closed surface		✓	✓	✓	✓	–	–	–
Model-related headboard with handle bar		–	–	–	–	–	–	–
Model-related footboard with handle bar		–	–	–	–	–	–	–
Comodo with round posts		–	–	–	–	–	–	–
Primero with handle bar on head and footboard		–	–	–	–	–	–	–
Prinzino with handle bar on head and footboard		–	–	–	–	–	–	–
Primus closed-surface headboard/ footboard with handle bar		–	–	–	–	–	–	–
Primus-Low closed-surface headboard/ footboard with handle bar		–	–	–	–	–	–	–
Nobla		–	–	–	–	–	–	–
Nobla headboard, Nobla mala footboard ³⁾		–	–	–	–	–	–	–
Decor								
Natural Beech ²⁾		✓	✓	✓	✓	–	–	–
Royal Maple ²⁾		–	–	–	–	–	–	–
Havana Cherry ²⁾		–	–	–	–	–	–	–
Lindberg Oak		–	–	–	–	–	–	–
Bella Noce Choco ²⁾		–	–	–	–	–	–	–

Subject to technical changes without notice ✓ = Standard O = Available as an option – = Not available ³⁾Footboard also available as the lower Primus-Low version

²⁾Available for Relax senior bed ³⁾Also available as upholstered version

Model		Regia	Regia partner	Inovia	Westfalia IV	Westfalia Klassik	Lenus	Gigant	Allura II 100 120
Adjustments									
Height	Electric	✓	✓	✓	✓	✓	✓	✓	✓
Backrest	Electric	✓	✓	✓	✓	✓	✓	✓	✓
Thigh rest	Electric	✓	✓	✓	✓	✓	✓	✓	✓
Locking feature	Selective	✓	✓	✓	✓	✓	✓	✓	✓
Locking feature	Central	–	–	–	–	–	–	–	–
Wireless handset		–	–	–	–	–	–	–	–
Cabled handset		✓	✓	✓	✓	✓	✓	✓	✓
24-volt drive system		✓	–	✓	✓	✓	0	–	✓
Econ drive system		–	–	–	–	–	–	–	–
Height adjustment range									
from – to	cm	26-80	26-80	22-77	35-80	35-80	15-80	40-80	30-80
4-section mattress base									
Standard	cm	90 x 200	2 x 90 x 200	90 x 200 100 x 200	90 x 180 190 200	90 x 200	90 x 200	120 x 200	100 120 x 200
Special mattress-base widths		100	–	–	100	–	–	–	–
Metal slats		✓	✓	✓	–	✓	✓	–	–
Wire mesh mattress base		–	–	–	–	–	–	✓	✓
Sprung wooden slats		–	–	–	✓	–	–	–	–
Comfort mattress base		0	0	0	–	–	0	–	–
One-piece		✓	✓	✓	–	✓	–	✓	–
Multi-section		–	–	–	✓	–	✓	–	✓
Adjustable mattress base tilt		✓	✓	✓	–	–	✓	✓	✓
Sliding backrest		✓	✓	✓	–	✓	✓	–	–
Ground clearance for lifting equipment		✓	✓	✓	✓	✓	✓	✓	✓
Castor locking									
Central		✓	✓	–	–	✓	–	–	–
Per axle		–	–	✓	–	–	✓	–	–
Single		–	–	–	✓	–	–	✓	✓
Castor diameter	mm	50	50	100	125	100	50	100	100
External dimensions and weight									
Length		210	210	220	212	212	237	222	222
Width		101	196	105 115	100	101	99	135	111 131
Net weight	kg	135	270	135 150	112	132	123	159	118 128
Safe working load									
	kg	250	2 x 225	225	185	200	225	350	250
Wooden surround									
Standard		–	–	–	✓ (90 cm only)	✓	–	–	–
Model-related headboard, closed surface		–	–	✓	–	–	✓	✓	–
Model-related footboard, closed surface		–	–	0	–	–	✓	✓	–
Model-related headboard with handle bar		–	–	0	–	–	–	–	✓
Model-related footboard with handle bar		–	–	✓	–	–	–	–	✓
Comodo with round posts		–	–	–	0	0	–	–	–
Primero with handle bar on head and footboard		–	–	–	0 (90 cm only)	0	–	–	–
Prinzino with handle bar on head and footboard		0	–	–	–	–	–	–	–
Primus closed-surface headboard/ footboard with handle bar		0 ¹⁾	–	–	–	–	–	–	–
Primus-Low closed-surface headboard/ footboard with handle bar		0	–	–	–	–	–	–	–
Nobla		0	0	–	–	–	–	–	–
Nobla headboard, Nobla mala footboard ⁴⁾		0	–	–	–	–	–	–	–
Decor									
Natural Beech ²⁾		✓	✓	✓	✓	✓	✓	✓	✓
Royal Maple ²⁾		0	–	–	–	–	–	–	–
Havana Cherry ²⁾		0	0	0	–	–	0	–	–
Lindberg Oak		0	–	–	–	–	–	–	–
Bella Noce Choco ²⁾		–	–	–	–	–	–	–	–

Subject to technical changes without notice ✓ = Standard 0 = Available as an option – = Not available ¹⁾Footboard also available as the lower Primus-Low version

²⁾Available for Relax senior bed ³⁾Also available as upholstered version

Technical data

Model	Dali	Dali wash	Dali low-econ	Dali low-entry	Lippe IV	Lippe IV washable	Lippe 120/140
Accessories							
Adaptable safety sides	0	–	0	–	0	–	–
Patient lifting pole including triangular handle	✓	✓	✓	✓	0	0	0
Clamp-on extra-high safety side rail	0	0	0	–	–	–	–
Standing aid	0	–	0	–	0	0	0
Bed extension 20 cm	0	–	0	–	–	–	–
Linen holder	–	–	–	–	–	–	–
Full-length safety side	✓	✓	✓	✓	–	–	–
Raised mattress-clamp handles (approx. 19 cm)	–	–	–	–	0	0	0
Upholstered side panels	–	–	–	–	–	–	–
Filler piece	0	0	0	0	–	–	–
Transmitter holder for handset	–	–	0	–	0	0	–
Handset extension cable	–	–	–	–	0	0	–
Combinable safety side	–	–	–	–	–	–	–
Transportation and storage aid	✓	✓	✓	✓	0	✓	–
Reading lamp	0	0	0	0	0	0	0
Lippe side protection	–	–	–	–	–	–	–
Ratchet mechanism for lower leg rest	0 ⁴⁾	✓	0	✓	✓	✓	✓
Foam leather padding, foam leather cover	0	0	0	0	–	–	–
Tray (90 cm width)	0	0	0	0	–	–	–
Softcovers	0	0	–	–	–	–	–
Telescopic safety side	–	–	–	–	–	–	–
Carry bag for DSG bars	0	0	0	0	–	–	–
Under bed light	–	–	0	–	–	–	0
Widening rail 5 cm	–	–	–	–	0	–	–
Wall spacer	–	–	–	–	–	–	–
Wall deflection rollers	0	0	0	0	–	–	–
Additional cladding (wooden surround)	0	–	0	0	–	–	–

Model	Regia	Regia partner	Inovia	Westfalia IV	Westfalia Klassik	Lenus	Gigant	Allura II 100 120
Accessories								
Adaptable safety sides	-	-	-	O	-	-	-	-
Patient lifting pole including triangular handle	O	O	O	✓	✓	O	✓	✓
Clamp-on extra-high safety side rail	-	-	-	-	-	-	O (DSG only)	-
Standing aid	-	-	-	O	-	-	-	O
Bed extension 20 cm	O (TSG 18 cm) O (DSG 20 cm)	O (on both sides only)	O (TSG 27 cm)	O	O	✓	O	O
Linen holder	O	-	-	-	-	-	-	-
Full-length safety side	O	O	✓	✓	✓	✓	✓	✓
Raised mattress-clamp handles (approx. 19 cm)	-	-	-	-	-	-	-	-
Upholstered side panels	O	O	-	-	-	-	-	-
Filler piece	O (DSG only)	O (DSG only)	O (DSG only)	O	O	O	O (DSG only)	O
Transmitter holder for handset	O	-	-	O	O	O	-	-
Handset extension cable	O	-	-	O	O	O	-	-
Combinable safety side	-	-	-	-	-	-	O	-
Transportation and storage aid	-	-	-	-	-	-	-	✓
Reading lamp	O	O	O	O	O	O	O	O
Lippe safety side	-	-	-	-	-	-	-	-
Ratchet mechanism for lower leg rest	✓	✓	✓	✓	✓	✓	✓	✓
Foam leather padding, foam leather cover	O	O (TSG only)	O	O	O	O	O	O
Tray (90 cm width)	O (DSG only)	-	O (DSG only)	O	O	O	-	-
Softcovers	-	-	-	-	-	O	-	-
Telescopic safety side	O	O	O	-	-	-	-	-
Carry bag for DSG bars	O	O	O	O	O	O	O	O
Under bed light	O	-	-	-	-	-	-	O
Widening rail 5 cm	-	-	-	-	-	-	-	-
Wall spacer	O	O	-	-	-	-	-	-
Wall deflection rollers	O	-	O	O	O	-	-	O
Additional wooden surround (wooden panel)	-	-	-	-	-	O	-	O

Dali range




Suitable for / Medical indication

- Standard bed with a good blend of protection and comfort
- Suitable for practically all care cases
- Easy for relatives to operate

Special features

- With wireless or cabled handset
- Safety sides with Easy Click system
- Machine washable version
- 24-volt drive system dispenses with leakage current testing

Lippe range/Relax

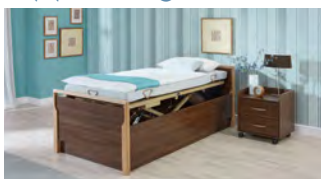

Suitable for / Medical indication

- Ideal for people who want to keep their own familiar bed
- Suitable for low to moderate levels of care, where no sitting function or bed castors are required

Special features

- Quickly assembled by just one person
- Small supporting frame for effortless installation in almost all bed frames
- Height adjustment from 29 - 74 cm
- Machine washable version

Regia/Inovia



Suitable for / Medical indication

- Optimal solution for all care requirements
- Ideal for patient mobilisation, thanks to split safety sides and a large height-adjustment range

Special features

- Low-height bed for fall prevention
- Comfort and elegance for private customers
- Regia also available as a partner bed
- Choice of homelike decors, fabrics and faux leather

Westfalia range



Suitable for / Medical indication

- The ideal choice for durable, robust use in everyday care
- Particularly suitable for people who are still mobile and are able to sit in a chair

Special features

- Care beds with stable scissor-type frame
- Attractive head and footboard designs
- Selectively lockable handset
- Cost-saving 24-volt drive system

Lenus




Suitable for / Medical indication

- Restless residents with low risk perception
- Persons with dementia and a tendency to wander

Special features

- Low-height bed for fall prevention
- Easy transfer for wheelchair users
- Comfortable sitting position for mealtimes and leisure activities

Heavy duty beds

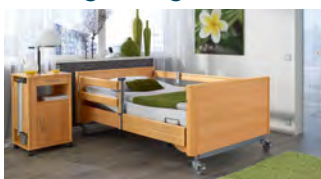

Suitable for / Medical indication

- Particularly suitable for heavy and obese residents
- All care requirements are met

Special features

- High safe working load, up to 350 kg
- Mattress base widths up to 120 cm
- Better quality of life for heavier residents

Bedside cabinets and side tables



Suitable for / Medical indication

- Recommended for residents who cannot leave the bed to eat meals
- Ideal for use with low-height beds (Varus model)

Special features

- Storage space for personal belongings and care utensils
- Ideal for drinks and meals taken in bed
- Convenient bookrests
- Good manoeuvrability for flexible use

Mattresses, transport chair and furniture



Suitable for / Medical indication

- Mattresses suitable for all Burmeier beds
- Help prevent bedsores, however there is no certified grade of effectiveness

Special features

- Customised furnishings for care rooms
- Easily accessible storage space
- Safe and comfortable transfer in the Ravello transport chair

With care equipment numbers	Very easy assembly Easy Click	Low-height bed 	Machine washable 	Bed extension 	Reverse-Trendelenburg position 	Werdenfeller approach
-----------------------------	---	--------------------	----------------------	-------------------	------------------------------------	---------------------------

With care equipment numbers	High safe working load 	Machine washable 	Large height adjustment range
-----------------------------	----------------------------	----------------------	-----------------------------------

Easy Switch * * only for Regia	Low-height bed 	High safe working load 	Bed extension 	Large height adjustment range 	Reverse-Trendelenburg position 	Werdenfeller approach
--	--------------------	----------------------------	-------------------	-----------------------------------	------------------------------------	---------------------------

With care equipment numbers	High safe working load 	Bed extension 	Large height adjustment range
-----------------------------	----------------------------	-------------------	-----------------------------------

Low-height bed 	High safe working load 	Bed extension 	Large height adjustment range 	Reverse-Trendelenburg position
--------------------	----------------------------	-------------------	-----------------------------------	------------------------------------

With care equipment numbers	Very easy assembly * Easy Click * only for Allura	High safe working load 	Bed extension 	Reverse-Trendelenburg position
-----------------------------	--	----------------------------	-------------------	------------------------------------

With care equipment numbers

Dali range



Robert Spencer,
Branch Manager,
medical supplies store

“Dali – when speed and reliability are called for.

The Dali range offers a tried-and-tested combination of particularly reliable product characteristics and quick delivery times.

It is quite simply the absolute standard on the market!”



Dave Johnson,
rehabilitation clinic
technician

“Dali wash – optimum hygiene in the battle against germs.

Machine reprocessing of care beds saves time and money.

And protecting users and carers from multi-resistant pathogens is paramount.”



James Borrow,
medical product consultant

“Dali low-entry – the low-height bed for stringent requirements.

With the low-height Dali low-entry, I have the ideal bed to offer residents who are particularly at risk of falling.”



Dali

Quick assembly and reliable operation

- ✓ Safe working load of 185 kg
- ✓ Maximum weight of resident: 155 kg
- ✓ Height adjustment range: 40-80 cm
- ✓ 24-volt drive system
- ✓ No leakage current measurements during expected service life
- ✓ Reverse-Trendelenburg position
- ✓ Tool-free safety side installation with Easy Click system
- ✓ Wireless or cabled handset



Dali wash

The bed for optimal hygiene

- ✓ Safe working load of 185 kg
- ✓ Maximum weight of resident: 155 kg
- ✓ Height adjustment range: 40-80 cm
- ✓ 24-volt drive system
- ✓ No leakage current measurements during expected service life
- ✓ Reverse-Trendelenburg position
- ✓ Machine washable for optimal hygiene
- ✓ Tool-free safety side installation with Easy Click system
- ✓ Wireless handset



Dali low-econ

The budget-oriented low-height bed

- ✓ Safe working load of 185 kg
- ✓ Maximum weight of resident: 155 kg
- ✓ Height adjustment range: 32-72 cm
- ✓ Econ drive system
- ✓ Reverse-Trendelenburg position
- ✓ Tool-free safety side installation with Easy Click system
- ✓ Selectively lockable, cabled handset




Dali low-entry

The low-height bed

- ✓ Safe working load of 175 kg
- ✓ Maximum weight of resident: 145 kg
- ✓ Height adjustment range: 23-63 cm
- ✓ 24-volt drive system
- ✓ No leakage current measurements during expected service life
- ✓ Reverse-Trendelenburg position
- ✓ Tool-free safety side installation with Easy Click system
- ✓ Cabled handset




Dali care bed

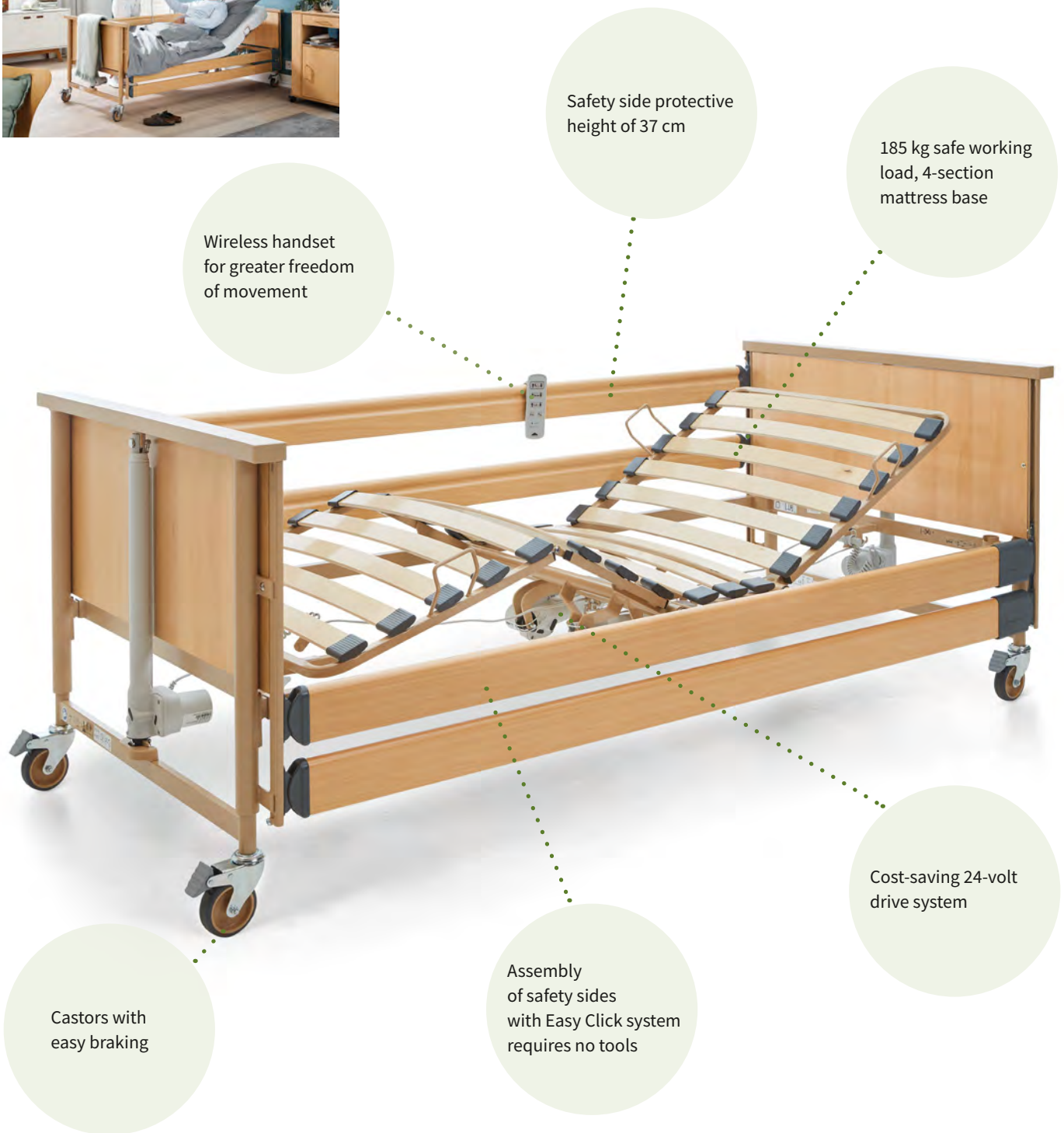

Advantages:

- Comfort, safety and ease of operation
- Very good price/performance ratio
- Easy and effortless attachment and removal with the Easy Click system
- Maximum flexibility – cabled handset or Bluetooth handset available

External dimensions/height adjustment range:

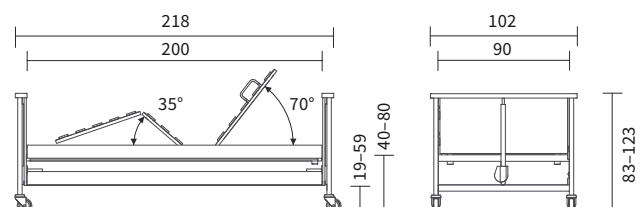

Design details



Reverse-Trendelenburg position and comfortable sitting position

- Makes breathing easier and reduces the risk of aspiration during meal times



Safety sides with Easy Click system

- Safety sides are attached and removed in seconds



Delivery and storage on transportation/storage aid

- Space-saving storage and safe transport



Handset

- Choice of wireless or cabled handset

Special features:



Further information:



- Product page



- Assembly video

Accessories and mattresses | Dali care bed



1 The adaptable safety side for Dali beds offers good protection and leaves enough space to get in and out of bed.



2 The clip-on safety side bar raises the side protection height by 18 cm.



A new safety-side height-extension rail will be available soon.



3 Standing aid.



4 5 Bed extension including safety side bars (also for retrofitting).




6 The filler piece can be fitted onto the full-length safety side to make it easier for residents to sit on the edge of the bed.



7 8 Homelike wooden surrounds for the head and footboards.



9 Reading lamp for easy insertion.

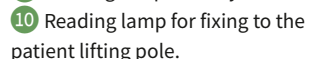

10 Reading lamp for fixing to the patient lifting pole.



11 Mattress piece for bed extension.



12 With the help of the optional ratchet mechanism, the lower leg rest can be manually raised.



13 Soft foam-leather covers protect the resident when in contact with the safety sides and when leaving the bed.



14 The tray is simply placed on top of the raised safety side bars.



15 Elegant Softcovers for head/footboards and safety sides – available in six different colours.



16 The bars of the safety sides can be transported and stored even more easily with the carry bag.



17 Wall deflection roller protect the bed and the walls from damage.



Vanilla

Bordeaux



Petrol

Dark blue



Lavender

Brown



18 Sky-fit care bed mattress.




19 Comfort-fit care bed mattress.




20 Basic-fit care bed mattress.

Technical data	Dali	
Net weight	97 kg	
External dimensions	102 x 218 cm	
Height adjustment range	40–80 cm	
Safe working load	185 kg	
Maximum resident weight	145 – 155 kg	
24-volt drive system	✓	
Standard features	Dali	
Frame colour	Topaz	
Decor	Natural Beech	
Mattress base size	90 x 200 cm	
Sprung wooden slat mattress base (mattress base model)	✓	
Telescopic safety side (DSG)	✓	
Type of handset (selectively lockable/wireless)	✓	
Mechanical CPR release	✓	
Castor diameter	100mm	
Castor locking	Single	
Patient lifting pole with triangular grab handle	✓	
Design features	Dali	Material no.
Dali with sprung wooden slat mattress base, wooden safety side, and Bluetooth handset		276349
Dali with metal slatted mattress base, wooden safety side, and Bluetooth handset		276354
Dali with sprung wooden slat mattress base, wooden safety side, and cabled handset		300673
Dali with metal slatted mattress base, wooden safety side, and cabled handset		300676
Dali with sprung wooden slat mattress base, wooden safety side, wood surround head/footboards, and cabled handset		300675
Dali with metal slatted mattress base, wooden safety side, wood surround head/footboards, and cabled handset		300677
Dali with sprung wooden slat mattress base, wooden safety side, wood surround head/footboards, and Bluetooth handset		276358
Dali with metal slatted mattress base, wooden safety side, wood surround head/footboards, and Bluetooth handset		276359
Special equipment and accessories	Dali	Material no.
1 Adaptable safety sides	Right Left	250138 250139
2 Clamp-on extra-high safety side rail	O	191367
3 Standing aid	O	175402
4 Bed extension for sprung wooden slat mattress base + safety side bars	O	291403
5 Bed extension for metal slatted mattress base + safety side bars	O	291739
6 Filler piece	O	258395
7 Wood surround as chassis cladding	1 piece	285163
8 Wood surrounds as chassis cladding	2 pieces per set	282157
9 Reading lamp for easy insertion	O	264877
10 Reading lamp for fixing to the patient lifting pole	O	289177
11 Mattress piece for bed extension	90 cm x 21 cm x 12 cm	166715
12 Ratchet mechanism, electrical adjustment of lower leg rest (sprung wooden slat mattress base)	O	197091
13 Foam leather cover, DSG	1 side	225921
14 Tray for full-length safety sides (DSG)	O	267000
15 Softcover	Vanilla Bordeaux Petrol Dark blue Lavender Brown	287280 287280 287280 287280 287280 287280
16 Carry bag for safety side bars	O	273831
17 Wall deflection rollers	O	206752
Mattresses	Dali	Material no.
18 Sky-fit mattress	90 cm x 200 cm x 14 cm	283924
19 Comfort-fit mattress	90 cm x 200 cm x 14 cm	283922
20 Basic-fit mattress	90 cm x 200 cm x 12 cm	261477

✓ = Standard feature O= Available as an option

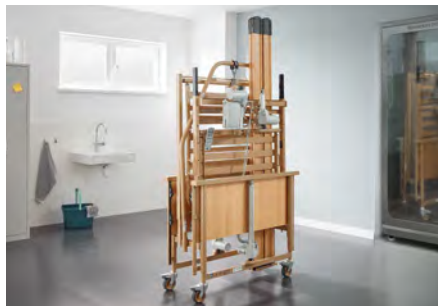


Dali wash care bed

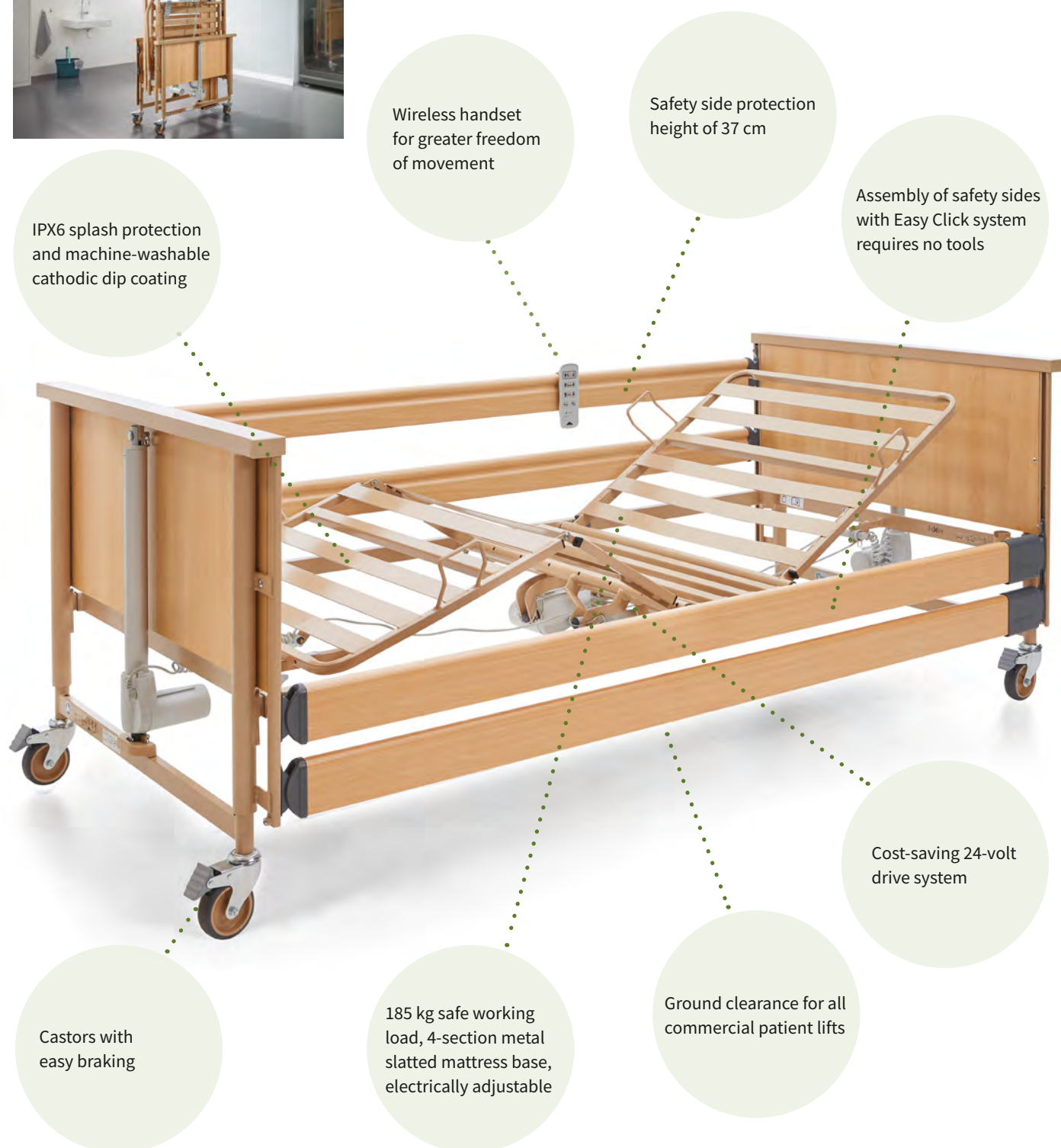

Advantages:

- Designed to allow cleaning in automated washing systems
- Can be reprocessed without removing the drive system, thanks to the high protection category: IPX6
- Corrosion protection with cathodic dip coating (CDC) and holes that allow residual fluid to drain away

External dimensions/height adjustment range:

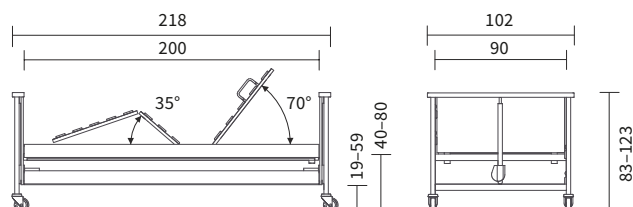

Design details



CDC coating

- Protects the bed from corrosion, inside and out



Drilled holes in the metal profiles

- These allow residual fluid to drain away



Plastic stoppers with drain holes

- These allow residual fluid to drain away



Sealed surfaces

- Ideal for safe and thorough cleaning



4-section mattress base

- With hygienic metal support bars

Special features:



Further information:



- Product page



- Assembly video



- Machine washable

Accessories and mattresses | Dali wash care bed



1 The clip-on safety side bar raises the side protection height by 18 cm.



A new safety-side height-extension rail will be available soon.



2 The filler piece can be fitted onto the full-length safety side to make it easier for residents to sit on the edge of the bed.



3 4 The reading lamp can be fitted to the patient lifting pole or into the connection sleeve.



5 Soft foam-leather covers protect the resident when in contact with the safety sides and when leaving the bed.



6 The tray is simply placed on top of the raised safety side bars.



7 Elegant Softcovers for head/footboards and safety sides – available in six different colours.



8 The bars of the safety sides can be transported and stored even more easily with the carry bag.



Vanilla



Bordeaux



Petrol



Dark blue



Lavender



Brown



9 Sky-fit care bed mattress.



10 Comfort-fit care bed mattress.



11 Basic-fit care bed mattress.

Technical data	Dali wash	
Net weight	97 kg	
External dimensions	102 x 218 cm	
Height adjustment range	40–80 cm	
Safe working load	185 kg	
Maximum resident weight	155 kg	
24-volt drive system	✓	
Prepared for use of Dali-Lock app	✓	
Standard features	Dali wash	
Frame colour	Topaz	
Decor	Natural Beech	
Mattress base size	90 x 200 cm	
Metal slatted mattress base (mattress base model)	✓	
Safety side	DSG	
Type of handset (selectively lockable/wireless)	✓	
Mechanical CPR release	✓	
Castor diameter	100mm	
Castor locking	Single	
Ratchet mechanism, electrical adjustment of lower leg rest for sprung wooden slat mattress base	✓	
Patient lifting pole with triangular grab handle	✓	
Design features	Dali wash	Material no.
Dali wash with metal slatted mattress base, wooden safety side, and Bluetooth handset		278060
Special equipment and accessories	Dali wash	Material no.
1 Clamp-on extra-high safety side rail	O	191367
2 Filler piece	O	258395
3 Reading lamp for easy insertion	O	264877
4 Reading lamp for fixing to the patient lifting pole	O	289177
5 Foam leather cover, DSG	1 side	225921
6 Tray for full-length safety sides (DSG)	O	267000
7 Softcover	Vanilla Bordeaux Petrol Dark blue Lavender Brown	287280 287280 287280 287280 287280 287280
8 Carry bag for safety side bars	O	273831
Mattresses	Dali wash	Material no.
9 Sky-fit mattress	90 cm x 200 cm x 14 cm	283924
10 Comfort-fit mattress	90 cm x 200 cm x 14 cm	283922
11 Basic-fit mattress	90 cm x 200 cm x 12 cm	261477




Dali low-econ low-height bed



Advantages:

- Combines many strengths and offers an excellent price/performance ratio
- With a height adjustment range from 32 to 72 cm, its lowest position protects residents from injuries caused by falls
- It also allows taller people to maintain a back-friendly, upright posture when working at the bed

External dimensions/height adjustment range:

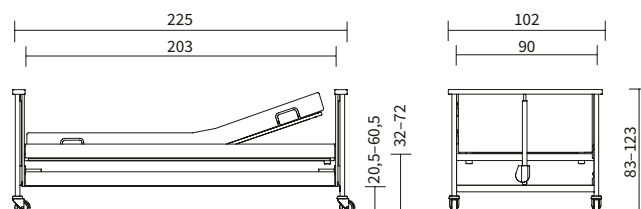

Design details



Mattress base

- With sprung wooden slats



Compact delivery package and storage

- On transportation and storage aid




Safe and reliable Econ 230-volt drive system.



Selectively lockable handset

Special features:

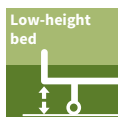

Further information:



- Product page

Accessories und mattresses | Dali low-econ



1 The adaptable safety side for Dali beds offers good protection and leaves enough space to get in and out of bed.



2 The clip-on safety side bar raises the side protection height by 18 cm.



3 Standing aid.



4 Bed extension including safety side bars (also for retrofitting).



5 The metal safety side is particularly robust and easy to clean.



6 The filler piece can be fitted onto the full-length safety side to make it easier for residents to sit on the edge of the bed.



7 Flexible transmitter holder for the handset.



8 Homelike wooden surrounds for the head and footboards.



9 10 The reading lamp can be fitted to the patient lifting pole or into the connection sleeve.



11 Mattress piece for bed extension.



12 With the help of the optional ratchet mechanism, the lower leg rest can be manually raised.



13 Soft foam-leather covers protect the resident when in contact with the safety sides and when leaving the bed.



14 The tray is simply placed on top of the raised safety side bars.



15 The bars of the safety sides can be transported and stored even more easily with the carry bag.



16 Wall deflection rollers protect the bed and the walls from damage.



17 Sky-fit care bed mattress.



18 Comfort-fit care bed mattress.



19 Basic-fit care bed mattress.

Technical data		Dali low-econ	
Net weight		97 kg	
External dimensions		102 x 225 cm	
Height adjustment range		32–72 cm	
Safe working load		185 kg	
Maximum resident weight		145 – 155 kg	
Econ 230-volt drive system		✓	
Standard features		Dali low-econ	
Frame colour		Topaz	
Decor		Natural Beech	
Mattress base size		90 x 200 cm	
Sprung wooden slat mattress base (mattress base model)		✓	
Telescopic safety side (DSG)		✓	
Type of handset (selectively lockable)		✓	
Mechanical CPR release		✓	
Castor diameter		100mm	
Castor locking		Single	
Patient lifting pole with triangular grab handle		✓	
Design features		Dali low-econ	Material no.
Dali low-econ with sprung wooden slat mattress base and wooden safety side			283952
Dali low-econ with sprung wooden slat mattress base and wooden safety side with wood surround head/footboards			283951
Special equipment and accessories		Dali low-econ	Material no.
1 Adaptable safety sides		Right	250138
		Left	250139
2 Clamp-on extra-high safety side rail		O	191367
3 Standing aid		O	175402
4 Bed extension for sprung wooden slat mattress base + safety side bars		O	1 x 266002
			4 x 271363
5 Full-length metal safety side		1 bar	271368
6 Filler piece		O	258395
7 Transmitter holder for handset		O	199137
8 Wood surrounds as chassis cladding		2 pieces per set	282159
9 Reading lamp for easy insertion		O	264877
10 Reading lamp for fixing to the patient lifting pole		O	1 x 191879
			1 x 174231
11 Mattress piece for bed extension		O	166715
12 Ratchet mechanism, electrical adjustment of lower leg rest		O	197091
13 Foam leather cover, DSG		1 side	225921
14 Tray for full-length safety sides (DSG)		O	174296
15 Carry bag for safety side bars		O	273831
16 Wall deflection roller		O	206752
Mattresses		Dali low-econ	Material no.
17 Sky-fit mattress		90 cm x 200 cm x 14 cm	283924
18 Comfort-fit mattress		90 cm x 200 cm x 14 cm	283922
19 Basic-fit mattress		90 cm x 200 cm x 12 cm	261477



Dali low-entry low-height bed

Low-level entry thanks to height adjustment from 23 to 63 cm

175 kg safe working load, 4-section metal slatted mattress base, electrically adjustable

Assembly of safety sides with Easy Click system requires no tools



Ratchet mechanism for adjusting the lower leg rest


Reverse-Trendelenburg position can be set

Cost-saving 24-volt drive system

Advantages:

- Helps prevent falls, without the use of liberty-depriving measures
- With its height adjustment range of 23 to 63 cm, it is close to the floor when at its lowest position
- Reduces the risk of injury should falls occur
- This makes it easier to decide not to use safety sides

External dimensions/height adjustment range:

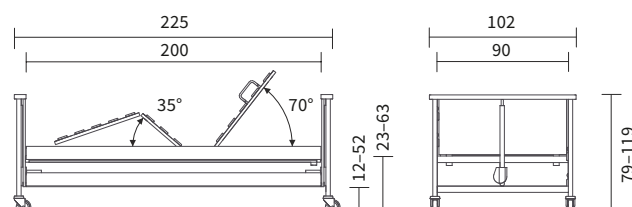

Design details



Reverse-Trendelenburg position

- Can help reduce reflux during the night



Electronic SwitchMode power supply unit

- Integrated immediately downstream of the mains plug



Connector socket with safety clip


- For connecting the electronic SwitchMode power supply unit
- The safety clip between the connector of the power supply unit and the socket on the bedframe ensures the reliable transfer of electricity and signals
- On all care beds with a 24-volt drive system



Transportation and storage aid

- For compact delivery and storage

Special features:

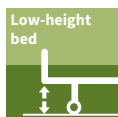

Further information:



- Product page



- Assembly video



- Interview with
RA Hackstein
in the FORUM

Accessories und mattresses | Dali low-entry



1 The filler piece can be fitted onto the full-length safety side to make it easier for residents to sit on the edge of the bed.



2 3 Wood surrounds as chassis cladding.



4 5 The reading lamp can be fitted to the patient lifting pole or into the connection sleeve.



6 Soft foam-leather covers protect the resident when in contact with the safety sides and when leaving the bed.



7 The tray is simply placed on top of the raised safety side bars.



8 The bars of the safety sides can be transported and stored even more easily with the carry bag.



9 Wall deflection rollers protect the bed and the walls from damage.



A new safety-side height-extension rail will be available soon.



10 Sky-fit care bed mattress.



11 Comfort-fit care bed mattress.



12 Basic-fit care bed mattress.

Technical data	Dali low-entry	
Net weight	97 kg	
External dimensions	102 x 225 cm	
Height adjustment range	23–63 cm	
Safe working load	175 kg	
Maximum resident weight	145 kg	
24-volt drive system	✓	
Prepared for use of Dali-Lock app	✓	
Standard features	Dali low-entry	
Frame colour	Topaz	
Decor	Natural Beech	
Mattress base size	90 x 200 cm	
Metal slatted mattress base (mattress base model)	✓	
Safety side	DSG	
Type of handset	Selective	
Mechanical CPR release	✓	
Castor diameter	75mm	
Castor locking	Single	
Ratchet mechanism, electrical adjustment of lower leg rest for sprung wooden slat mattress base	✓	
Patient lifting pole with triangular grab handle	✓	
Design features	Dali low-entry	Material no.
Dali low-entry with metal slatted mattress base, wooden safety side, and cabled handset		300674
Special equipment and accessories	Dali low-entry	Material no.
① Filler piece	O	258395
② Wood surround as chassis cladding	1 piece	285164
③ Wood surrounds as chassis cladding	2 pieces per set	282159
④ Reading lamp for easy insertion	O	264877
⑤ Reading lamp for fixing to the patient lifting pole	O	289177
⑥ Foam leather cover, DSG	1 side	225921
⑦ Tray for full-length safety sides (DSG)	O	267000
⑧ Carry bag for safety side bars	O	273831
⑨ Wall deflection rollers	1 piece	206752
Mattresses	Dali low-entry	Material no.
⑩ Sky-fit mattress	90 cm x 200 cm x 14 cm	283924
⑪ Comfort-fit mattress	90 cm x 200 cm x 14 cm	283922
⑫ Basic-fit mattress	90 cm x 200 cm x 12 cm	261477

Lippe range/Relax



Robert Spencer,
Branch Manager,
medical supplies store

“Lippe – the intelligent option for my customers.

Bed inserts from the Lippe range have a particularly space-saving supporting frame and fit into practically any bed surround. I can offer these to almost all customers who want to keep their own familiar bed. The assembly work can be done by one person.”



Angelika Herder,
customer

“Relax – new design for my bedroom.

Actually, I chose a Lippe bed insert because I wanted to keep my old bed. But then I felt like going for a new, more modern look. I really like the Relax bed surround.”

Lippe IV



The height-adjustable bed insert

- ✓ Safe working load of 200 kg
- ✓ Maximum weight of resident: 160 kg
- ✓ Height adjustment range: 29-74 cm
- ✓ 24-volt drive system
- ✓ No leakage current measurements during expected service life
- ✓ Mattress base dimensions of 80, 90 and 100 cm in width and 190 and 200 cm in length
- ✓ Space saving supporting frame measuring only 69 x 120 cm
- ✓ Secure and stable thanks to feet with floor-level adjustment screws

Lippe IV washable



The bed insert for optimal hygiene

- ✓ Safe working load of 200 kg
- ✓ Maximum weight of resident: 160 kg
- ✓ Height adjustment range: 29-74 cm
- ✓ 24-volt drive system
- ✓ No leakage current measurements during expected service life
- ✓ Machine washable for optimal hygiene
- ✓ Mattress base size 90 x 190 cm and 90 x 200 cm
- ✓ Selectively lockable handset
- ✓ Secure and stable thanks to feet with floor-level adjustment screws


Lippe 120/140



More space and comfort whatever the circumstances

- ✓ Safe working load of 250 kg
- ✓ Maximum weight of resident: 210 kg
- ✓ Height adjustment range: 29-74 cm
- ✓ 24-volt drive system
- ✓ No leakage current measurements during expected service life
- ✓ Selectively lockable handset
- ✓ Secure and stable thanks to feet with floor-level adjustment screws

Relax

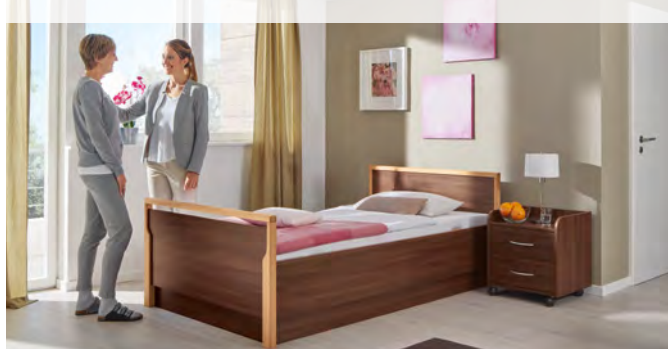

The combinable, home-comfort bed surround

- ✓ Mattress base dimensions of 90, 100, 120 and 140 cm
- ✓ Lengths of 200 and 220 cm
- ✓ Headboard and footboard frames made of solid beech
- ✓ Panels and bed sides in Natural Beech, Royal Maple, Havana Cherry or Bella Noce Choco
- ✓ Optional wooden cover to bridge the middle gap of a double bed



Lippe IV bed insert

200 kg safe working load, 4-section mattress base, electrically adjustable

Handset with selective locking function as a standard feature

Mattress clamp handles can be adjusted to widen the mattress base by 5 cm

Ratchet mechanism for adjusting the lower leg rest

Cost-saving 24-volt drive system

Mattress clamp handle, mattress retainer bar at foot end allows the mattress base to be extended by 5 cm


Small supporting frame measuring only 69 x 120 cm for effortless installation

Feet with floor-level adjustment screws and fixing lugs

Advantages:

- Can be installed in an existing bedframe
- Offers all the advantages of an electrically adjustable care bed
- The large height-adjustment range makes getting out of bed easy and allows carers to work at an ergonomically ideal height

External dimensions/height adjustment range:

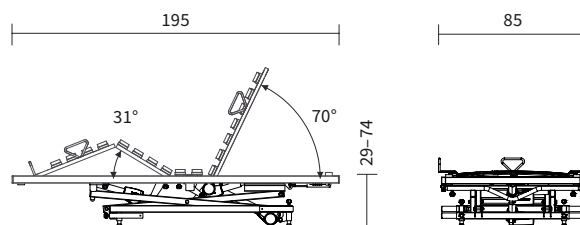

Design details



Mattress clamp handles

- For lengthening and widening by 5 cm respectively, detachable



Electronic SwitchMode power supply unit

- Integrated immediately downstream of the mains plug




Selectively lockable handset



Feet


- With floor-level adjustment screws and fixing lugs

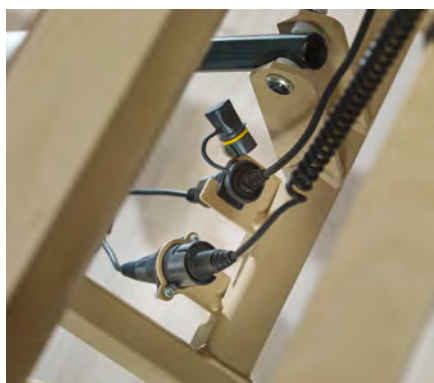


Connection socket for the plug

- For connecting the electronic SwitchMode power supply unit

Special features:

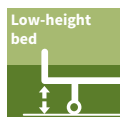

Low-height bed

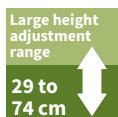

Large height adjustment range
29 to 74 cm



High safe working load
200 kg



24 volt system

Further information:



- [Product page](#)



- [Assembly video](#)

Accessories and mattresses | Lippe IV bed insert



1 Adaptable safety sides.



2 Adjustable feet for raising the lowest position (+ 7 cm).



3 Standing aid.



4 Patient lifting pole.



5 Flexible transmitter holder for the handset.



6 Extension cable for the handset (2.40 m).



7 Transportation and storage aid with steerable castors.



8 9 Reading lamps.



10 Lippe safety side (also for retrofitting).



11 Widening rail, 5 cm.



12 Sky-fit care bed mattress.



13 Comfort-fit care bed mattress.




14 Basic-fit care bed mattress.




15 Cloud-fit heavy-duty mattress.

Technical data	Lippe IV	
Net weight	80 kg	
External dimensions	85 x 185 cm 85 x 195 cm 95 x 185 cm 95 x 195 cm 75 x 185 cm 75 x 195 cm	
Height adjustment range	29–74 cm	
Safe working load	200 kg	
Maximum resident weight	160 kg	
24-volt drive system	✓	
Standard features	Lippe IV	
Frame colour	Topaz	
Mattress base size	90 x 200 cm	
Sprung wooden slat mattress base (mattress base model)	✓	
Type of handset (selectively lockable/wireless)	Selective	
Mechanical CPR release	✓	
Ratchet mechanism, electrical adjustment of lower leg rest	✓	
Design features	Lippe IV	Material no.
Lippe IV 90 x 190 cm		288381
Lippe IV 90 x 200 cm		288382
Lippe IV 100 x 190 cm		288383
Lippe IV 100 x 200 cm		288384
Lippe IV 80 x 190 cm		288378
Lippe IV 80 x 200 cm		288380
Special equipment and accessories	Lippe IV	Material no.
1 Adaptable safety sides	Right Left	250138 250139
2 Adjustable feet	1 piece	180431
3 Standing aid	O	175402
4 Patient lifting pole with triangular grab handle	O	146602
5 Transmitter holder for handset	O	199137
6 Handset extension lead (2.4 m)	O	178986
7 Transportation and storage aid	O	289524
8 Reading lamp for easy insertion	O	264877
9 Reading lamp for fixing to the patient lifting pole	O	289177
10 Lippe safety side	O	180457
11 Widening rail to widen mattress base by 5 cm	O	183505
Mattresses	Lippe IV	Material no.
12 Sky-fit mattress	90 cm x 200 cm x 14 cm 100 cm x 200 cm x 14 cm	283924 283925
13 Comfort-fit mattress	90 cm x 200 cm x 14 cm 100 cm x 200 cm x 14 cm	283922 283923
14 Basic-fit mattress	80 cm x 190 cm x 12 cm 80 cm x 200 cm x 12 cm 90 cm x 190 cm x 12 cm 90 cm x 200 cm x 12 cm 90 cm x 220 cm x 12 cm 100 cm x 190 cm x 12 cm 100 cm x 200 cm x 12 cm 100 cm x 220 cm x 12 cm	261479 261481 261483 261477 261484 261485 261486 261487
15 Cloud-fit mattress	100 cm x 200 cm x 16 cm	300900

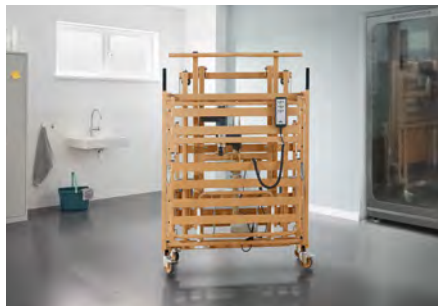


Lippe IV washable bed insert

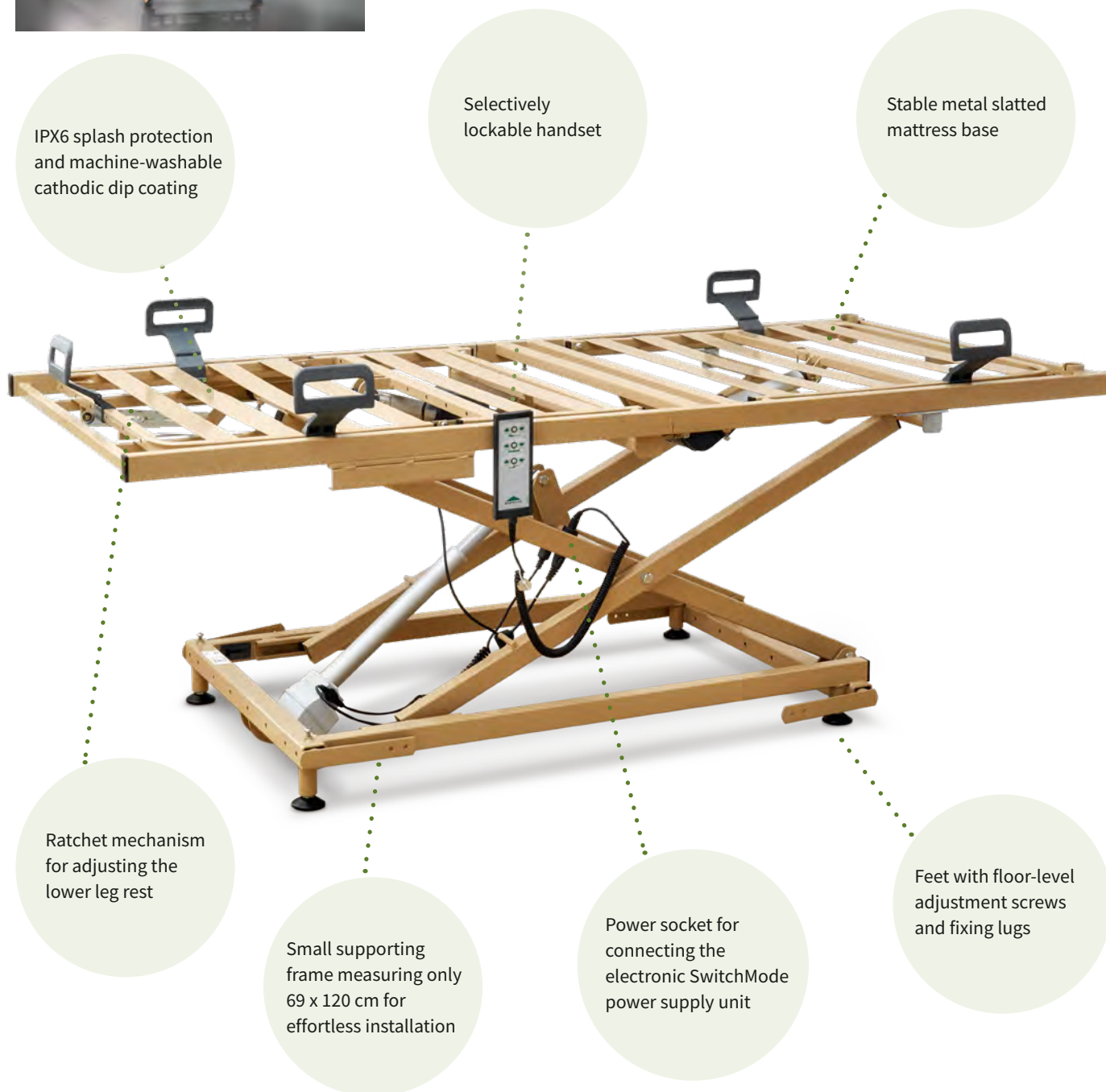

Advantages:

- Designed to allow cleaning in automated washing systems
- Can be reprocessed without removing the drive system, thanks to the high protection category: IPX6
- Corrosion protection with cathodic dip coating (CDC) and holes that allow residual fluid to drain away

External dimensions/height adjustment range:

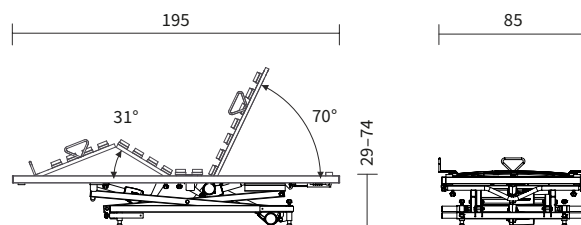

Design details



Mattress clamp handles

- For lengthening and widening by 5 cm respectively, detachable



Electronic SwitchMode power supply unit

- Integrated immediately downstream of the mains plug




Selectively lockable handset



Feet

- With floor-level adjustment screws and fixing lugs

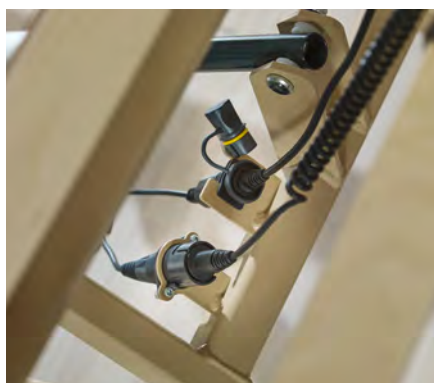

Connection socket for the plug



- For connecting the electronic SwitchMode power supply unit



Transportation and storage aid

- With steerable castors

Special features:

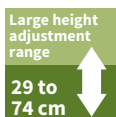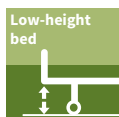

Further information:



- Product page



- Assembly video



- Machine washable

Accessories and mattresses | Lippe IV washable bed insert



1 Adjustable feet for raising the lowest position (+ 7 cm).



2 Patient lifting pole.



3 Standing aid.



4 Flexible transmitter holder for the handset.



5 Extension cable for the handset (2.40 m).



6 7 Reading lamps.



8 Sky-fit care bed mattress.



9 Comfort-fit care bed mattress.



10 Basic-fit care bed mattress.

Technical data	Lippe IV washable	
Net weight	80 kg	
External dimensions	85 x 185 cm 85 x 195 cm	
Height adjustment range	29–74 cm	
Safe working load	200 kg	
Maximum resident weight	160 kg	
24-volt drive system	✓	
Standard features	Lippe IV washable	
Frame colour	Topaz	
Mattress base size	90 x 190 cm 90 x 200 cm	
Metal slatted mattress base (mattress base model)	✓	
Type of handset (selectively lockable/wireless)	Selective	
Mechanical CPR release	✓	
Ratchet mechanism, electrical adjustment of lower leg rest	✓	
Design features	Lippe IV washable	Material no.
Lippe IV washable 90 x 190 cm		287092
Lippe IV washable 90 x 200 cm		287093
Special equipment and accessories	Lippe IV washable	Material no.
① Adjustable feet	1 piece	180431
② Patient lifting pole with triangular grab handle	O	146602
③ Standing aid	O	175402
④ Transmitter holder for handset	O	199137
⑤ Handset extension lead (2.4 m)	O	178986
⑥ Reading lamp for easy insertion	O	264877
⑦ Reading lamp for fixing to the patient lifting pole	O	289177
Mattresses	Lippe IV washable	Material no.
⑧ Sky-fit mattress	90 cm x 200 cm x 14 cm	283924
⑨ Comfort-fit mattress	90 cm x 200 cm x 14 cm	283922
⑩ Basic-fit mattress	90 cm x 200 cm x 12 cm 90 cm x 190 cm x 12 cm	261477 261483




Lippe 120/140 bed insert



Advantages:

- Offers plenty of space and comfort, with wider mattress bases measuring 120 and 140 cm
- With a high safe working load of 250 kg, it is a practical alternative to a heavy-duty bed
- Depending on the type of mattress and accessories used, it can accommodate residents who weigh more than 200 kg

External dimensions/height adjustment range:

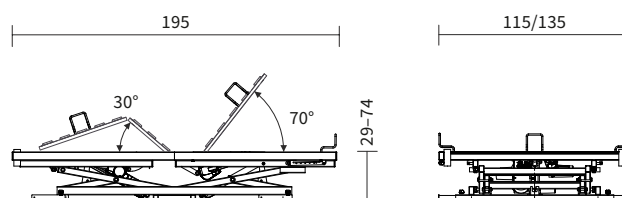

Design details



Feet

- With floor-level adjustment screws and fixing lugs



Ball bearing height adjustment



Electronic SwitchMode power supply unit

- SwitchMode integrated immediately downstream of the mains plug



Connection socket for the plug



- For connecting the electronic SwitchMode power supply unit



Low entry height

- 29 cm – makes getting in and out of bed easier

Special features:

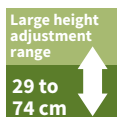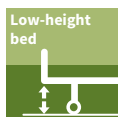

Further information:



- [Product page](#)



- [Assembly video](#)

Accessories and mattresses | Lippe 120 / 140 bed insert



1 Patient lifting pole.



2 Standing aid.



3 4 Mattress base extension
(also for retrofitting).



5 Extension cable for the handset
(2.40 m).



6 7 Reading lamps.



8 Mattress piece for mattress
base extension.



9 Under bed light.



10 Basic-fit care bed mattress.



11 Cloud-fit heavy-duty mattress.

Technical data	Lippe 120/140	
Net weight	129/138 kg	
External dimensions	115 x 195 cm 135 x 195 cm	
Height adjustment range	29–74 cm	
Safe working load	250 kg	
Maximum resident weight	210 kg	
24-volt drive system	✓	
Standard features	Lippe 120/140	
Frame colour	Topaz	
Mattress base size	120 x 200 cm 140 x 200 cm	
Metal slatted mattress base (mattress base model)	✓	
Type of handset (selectively lockable/wireless)	Selective	
Mechanical CPR release	✓	
Design features	Lippe 120/140	Material no.
Lippe 120 x 200 cm		288386
Lippe 140 x 200 cm		288388
Special equipment and accessories	Lippe 120/140	Material no.
① Patient lifting pole with triangular grab handle	O	146602
② Standing aid	O	244029
③ 20-cm bed length extension for 120-cm wide metal slatted mattress base	O	240285
④ 20-cm bed length extension for 140-cm wide metal slatted mattress base	O	240286
⑤ Handset extension cable (1.4 m)	O	238047
⑥ Reading lamp for easy insertion	O	264877
⑦ Reading lamp for fixing to the patient lifting pole	O	289177
⑧ Mattress piece for mattress base extension	120 cm x 21 cm x 16 cm 120 cm x 21 cm x 19 cm	241942 712167
⑨ Under bed light	O	249089
Mattress und mattress pieces	Lippe 120/140	Material no.
⑩ Basic-fit mattress	140 cm x 200 cm x 16 cm	261489
⑪ Cloud-fit mattress	120 cm x 200 cm x 16 cm 140 cm x 200 cm x 16 cm	300406 300407

Relax bed surround



Advantages:

- Wooden bed surround for home comfort. It can be combined with the Lippe range models to create an elegant bed
- Electrically adjustable slatted frame available
- Also suitable for people who are looking for a double bed with technical support
- Two bed surrounds can be combined to create a double bed



The homelike Relax bed frame is ideal for combining with a Lippe bed insert or a slatted frame as required.

Design details



Wooden frame in solid beech

- Panels and bed sides in Natural Beech decor



Wooden frame in solid beech

- Panels and bed sides in Royal Maple decor



Wooden frame in solid beech

- Panels and bed sides in Havana Cherry decor



Wooden frame in solid beech

- Panels and bed sides in Bella Noce Choco decor

Further information:



- [Product page](#)


Accessories | Relax bed surround




1 Adjustable safety sides combined with Lippe IV.




2 Floor-level sides.

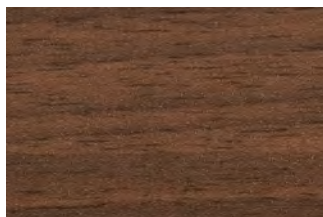


3 Bella Noce Choco decor.

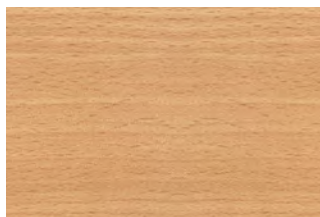

4 Natural Beech decor.

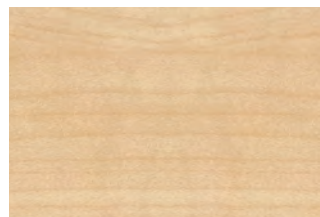

5 Royal Maple decor.

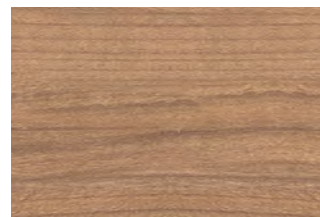

6 Havana Cherry decor.



7 Flexible transmitter holder for the handset, for use with Lippe IV.



8 9 Extension cable for the handset, for use with Lippe IV/ Lippe wash (2.40 m) or with Lippe 120/140 (1.40 m).



10 11 Motorised frame. Electrically adjustable backrest and leg rest (90 x 200 cm and 100 x 200 cm).



12 13 Reading lamps for use with Lippe IV, Lippe 120/140.



14 Wooden cover to join the gap between beds when used as a double bed.

Technical data	Relax	
External dimensions	100 x 209 cm	
	110 x 209 cm	
	130 x 209 cm	
	130 x 229 cm	
	150 x 209 cm	
	150 x 229 cm	
Standard features	Relax	
Mattress base size	90 x 200 cm	
	100 x 200 cm	
	120 x 200 cm	
	120 x 220 cm	
	140 x 200 cm	
	140 x 220 cm	
Design features	Relax	Material no.
Relax 90 x 200 cm		287200
Relax 100 x 200 cm		287200
Relax 120 x 200 cm		287200
Relax 120 x 220 cm		287200
Relax 140 x 200 cm		287200
Relax 140 x 220 cm		287200
Special equipment and accessories	Relax	Material no.
1 Adjustable safety sides for use with Lippe IV	Right Left	250138 250139
2 Floor-level sides	O	ZAHU004
3 Bella Noce Choco decor	O	ZHOD008
4 Natural Beech decor	O	ZHOD002
5 Havana Cherry decor	O	ZHOD005
6 Royal Maple decor	O	ZHOD004
7 Transmitter holder for handset, for use with Lippe IV	O	199137
8 Handset extension cable, for use with Lippe IV/Lippe wash	O	178986
9 Handset extension cable, for use with Lippe 120/140	O	238047
10 Slatted frame, electrically adjustable, 90 x 200 cm	O	289099
11 Slatted frame, electrically adjustable, 100 x 200 cm	O	289101
12 Reading lamp for easy insertion, for use with Lippe IV, Lippe 120/140	O	264877
13 Reading lamp for fixing to the patient lifting pole, for use with Lippe IV, Lippe 120/140	O	289177
14 Wooden cover for the middle gap, Natural Beech	O	217324

Regia/Inovia



James Borrow,
medical product consultant

“Regia – the bed for comfort and a homelike feel.

Many people in need of care, and their relatives, would like a level of comfort and style that goes beyond the standard. The Regia is an elegant model with a lot of options for creating an individual look. At the same time, it helps users to live independently.”



Sara and Robert McAllister,
customers

“Regia partner – the bed for couples to share a happy life together.

My husband and I often need care support. But we can’t imagine lying in two care beds that are separated from each other. With the Regia partner care double bed, we can still live our lives together.”



Regia

A bed to satisfy high individual expectations

- ✓ Safe working load of 225 kg
- ✓ Maximum weight of resident: 195 kg
- ✓ Height adjustment range: 26-80 cm
- ✓ 24-volt drive system
- ✓ No leakage current measurements during expected service life
- ✓ Reverse-Trendelenburg position
- ✓ Backrest retraction in accordance with the recommendations of the German Nurses Association (DBfK)
- ✓ Split telescopic safety side with Easy Switch, or full-length safety side
- ✓ Choice of attractive wood decors and fabric or leather upholstery



Regia partner

The double bed for a fulfilled partnership

- ✓ Large height adjustment range of 26 – 80 cm
- ✓ Two mattress bases measuring 90 x 200 cm each
- ✓ Safe working load of 225 kg per mattress base
- ✓ Backrest and leg rest individually adjustable
- ✓ Height adjustment and a reverse-Trendelenburg position of the bed corpus are always synchronous, so do not pose a pinching hazard for users
- ✓ Bed frame surrounds and safety sides can be flexibly adapted with the Easy Switch system



Inovia

The bed with innovative ideas

- ✓ Safe working load of 225 kg
- ✓ Maximum weight of resident: 195 kg
- ✓ Height adjustment range: 22-77 cm
- ✓ 24-volt drive system
- ✓ No leakage current measurements during expected service life
- ✓ Reverse-Trendelenburg position
- ✓ Backrest retraction in accordance with the recommendations of the German Nurses Association (DBfK)
- ✓ Optional telescopic split safety side
- ✓ Attractive wood decors to choose from



Regia low-height bed


**Easy
switch**



Advantages:

- Elegant care bed for discerning private customers
- Low-height bed with large height-adjustment range
- The flexible Easy Switch system with optional split safety sides protects the resident from fall injuries, without the use of liberty-depriving measures


External dimensions/height adjustment range:

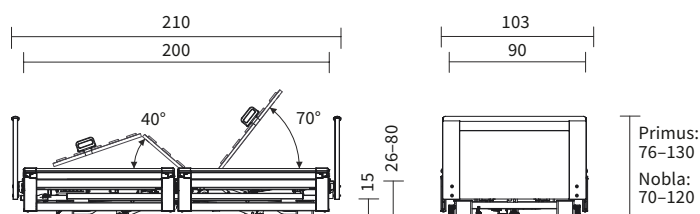



The homelike Regia low-height bed can be adapted to meet every comfort and protection need.

Special features:

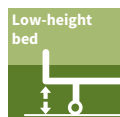

Further information:



- Product page
- This link will take you to the digital Regia configuration form.

Product video

Assembly video

Side elements | Regia care bed



1 Split safety sides at head and foot ends, telescopic (TSG).



2 3 Split safety side at head end, telescopic (TSG), and wooden side panels.



4 5 Split safety side at head end, telescopic (TSG), and fabric/faux leather side panels.



6 Wooden side panels.



7 Fabric/faux leather side panels.



8 Full-length safety sides on both sides (DSG) with Easy Click system.

This link will take you to the digital
Regia configuration form.



Technical data		Regia	
Net weight		135 kg	
External dimensions		103 x 210 cm 113 x 210 cm	
Height adjustment range		26–80 cm	
Safe working load		225 kg	
Maximum resident weight		195 kg	
24-volt drive system		✓	
Standard features		Regia	
Frame colour		Argentum	
Decor		Natural Beech	
Mattress base size		90 x 200 cm, 100 x 200 cm	
Metal slatted mattress base (mattress base model)		✓	
Ratchet mechanism, electrical adjustment of lower leg rest for metal slatted mattress base		✓	
Basic model		Regia	Material no.
Regia care bed 90 x 200 cm		Mattress base and chassis - 4-section mattress base with metal slats - Height adjustment range of 26 - 80 cm - Central braking system and concealed castors for a homelike touch	287900
Regia care bed 100 x 200 cm		The extra cost includes the 100-cm-wide mattress base	ZLFB002
Side elements		Regia	Material no.
1 Split safety sides at head and foot ends, telescopic (TSG)		- 2 x TSG 110 cm - 2 x TSG 90 cm - Can be attached at head or foot end as required	SEE_001
2 Split safety sides at head end, telescopic (TSG), and wooden side panels		- 2 x TSG at head end 110 cm - 2 x wooden side panels at foot end 90 cm - TSG 110 cm is fully compliant with the standards	SEE_004 SEA_001 SBU_001
3 Split safety side at head end, telescopic (TSG), and wooden side panels		- 2 x TSG at head end 90 cm - 2 x wooden side panels at foot end 110 cm - TSG 90 leaves plenty room for getting in and out of bed	SEE_004 SEA_002 SBU_001
4 Split safety sides at head end, telescopic (TSG), and fabric/faux leather side panels		- 2 x TSG at head end 110 cm - 2 x fabric/faux leather side panels at foot end 90 cm - TSG 110 cm is fully compliant with the standards	SEE_004 SEA_001 SBU_002
5 Split safety side at head end, telescopic (TSG), and fabric/faux leather side panels		- 2 x TSG at head end 90 cm - 2 x fabric/faux leather side panels at foot end 110 cm - TSG 90 leaves plenty room for getting in and out of bed	SEE_004 SEA_002 SBU_002
6 Wooden side panels		- 2 x wooden side panels 110 cm - 2 x wooden side panels 90 cm - Can be attached at head or foot end as required	SEE_003 SEA_001 SBU_001
7 Fabric/faux leather side panels		- 2 x fabric/faux leather side panels 110 cm - 2 x fabric/faux leather side panels 90 cm - Can be attached at head or foot end as required	SEE_003 SEA_001 SBU_002
8 Full-length safety sides on both sides (DSG)		- 4 x safety side bars 200 cm - 2 x side panels 200 cm	SEE_002

Headboards, footboards, decors | Regia care bed



1 Primus.



2 Primus low.



3 Prinzino.



4 Nobla.



5 Nobla mala.




6 Nobla upholstered.



7 Nobla mala upholstered.



8 Standard decor: Natural Beech.

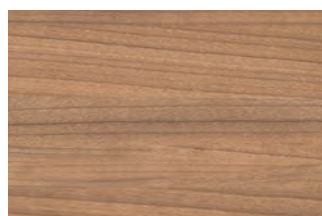

9 Special decor: Havana Cherry.



10 Special decor: Royal Maple.




11 Special decor Lindberg Oak.




12 Covering fabric: light green Tiba.



13 Covering fabric: red Tiba.

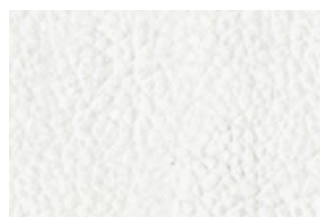

14 Faux leather, white.

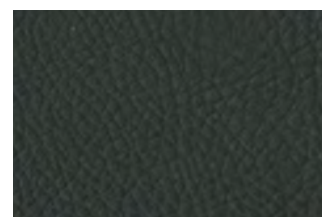


15 Faux leather, Anthracite.

This link will take you to the digital
Regia configuration form.



Headboards and footboards	Regia	Material no.
1 Primus	- 1 x headboard, closed, with solid wood bar - 1 x footboard with open handle and solid wood bar	KFH_022
2 Primus low Cannot be combined with full-length safety sides (DSG)	- 1 x headboard, closed, with solid wood bar - 1 x footboard with low-height open handle and solid wood bar for an unobstructed view of the room	KFH_023
3 Prinzino Not in Real Cherry decor	- 1 x headboard with open handle bar made of solid wood - 1 x footboard with open handle bar made of solid wood	KFH_004
4 Nobla Cannot be combined with full-length safety sides (DSG)	- Nobla headboard - Nobla footboard	KFH_011
5 Nobla mala Cannot be combined with full-length safety sides (DSG)	- Nobla headboard - Nobla mala low-height footboard	KFH_013
6 Nobla (upholstered) Cannot be combined with full-length safety sides (DSG)	- 1 x headboard, upholstered - 1 x footboard, upholstered - Fixed upholstery with non-removable fabric/faux leather	KFH_019
7 Nobla mala (upholstered), cannot be combined with full-length safety sides (DSG)	- 1 x headboard, upholstered - 1 x low-height footboard, upholstered - Fixed upholstery with non-removable fabric/faux leather	KFH_020
Wood decors	Regia	Material no.
8 Standard decor	Natural Beech R 24010	DHU_027
9 Special decor	Havana Cherry R 42006	DHU_024
10 Special decor	Royal Maple R 27001	DHU_022
11 Special decor	Lindberg Oak R 20021	DHU_044
Cover materials	Regia	Material no.
12 Covering fabric	Light green Tiba 6210	BSH_H006
13 Covering fabric	Red Tiba 3210	BSH_H001
Faux leather	Regia	Material no.
14 Faux leather	White 1-1142-197	BSH_K009
15 Faux leather	Anthracte 1-1142-395	BSH_K025

Accessories and mattresses | Regia care bed

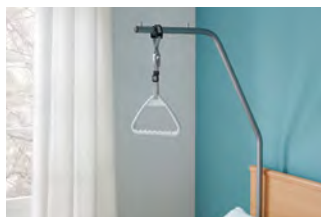

1 Patient lifting pole.



2 3 Bed extension.



4 Linen holder.



5 External battery.



6 Filler piece for full-length safety sides.




7 Transmitter holder for handset.
19 Trendelenburg handset.



8 Handset extension cable (1.40 m).



9 10 Comfort mattress base.

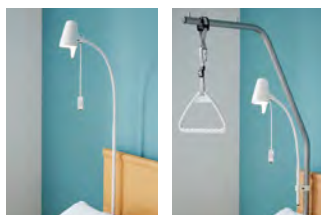

11 Reading lamp for easy insertion.
12 Reading lamp for fixing to the patient lifting pole.



13 Mattress piece for bed extension.



14 Foam leather cover, DSG.



15 16 Foam leather cover for telescopic safety sides (TSG).



17 Tray.



18 Carry bag for safety side bars.



20 Under bed light.



21 22 Wall spacer.



23 Wall deflection roller.



24 Sky-fit mattress.



25 Comfort-fit mattress.



26 Basic-fit mattress.

This link will take you to the digital Regia configuration form.



Special equipment and accessories	Regia	Material no.
1 Patient lifting pole with triangular grab handle		188223
2 Bed extension for metal slatted mattress base 90 and 100 cm with TSG	- 90 cm width - 100 cm width	ZLFL011
3 Bed extension for metal slatted mattress base with DSG	- 90 cm width - 100 cm width	ZLFL007
4 Linen holder	O	247350
5 External battery	O	275602
6 Filler piece for full-length safety sides	Dimensions (W x L x H): 57 x 12.5 x 5 cm - Colour: Beige	258395
7 Transmitter holder for handset	O	199137
8 Handset extension cable (1.40 m)	O	238047
9 Comfort mattress base		245702
10 Comfort mattress base for bed extension	O	257471
11 Reading lamp for easy insertion	O	264877
12 Reading lamp for fixing to the patient lifting pole	O	289177
13 Mattress piece for bed extension, 90 cm wide	- Extends the mattress when the bed extension is in use - Dimensions (W x L x H): 90 x 21 x 12 cm - Colour: White - With PU cover	166715
14 Foam leather cover, DSG	O	225921 (one element)
15 Foam leather cover for 90 cm telescopic safety sides (TSG)	O	733670 (one element)
16 Foam leather cover for 110 cm telescopic safety sides (TSG)	O	733669 (one element)
17 Tray 90 cm	O	267000
18 Carry bag for safety side bars (DSG)	O	273831
19 Trendelenburg handset	O	ZHS_003
20 Under bed light	O	277987
21 Wall spacer at head end	O	262024
22 Wall spacer at side	O	262000
23 Wall deflection roller	O	288305
Mattresses	Regia	Material no.
24 Sky-fit mattress	- 90 cm x 200 cm x 14 cm - 100 cm x 200 cm x 14 cm	283924 283925
25 Comfort-fit mattress	- 90 cm x 200 cm x 14 cm - 100 cm x 200 cm x 14 cm	283922 283923
26 Basic-fit mattress	- 90 cm x 200 cm x 12 cm - 90 cm x 220 cm x 12 cm - 100 cm x 200 cm x 12 cm - 100 cm x 220 cm x 12 cm	261477 261484 261486 261487



Regia partner double bed

**Easy
Switch**

Height adjustment and tilting to a reverse-Trendelenburg position are always synchronous, so do not pose a pinching hazard

Backrest and leg rests of the mattress base individually adjustable

Optional comfort mattress base made of free-floating elements can be placed on the metal slats



Safety sides, headboards, footboards and panels with the Easy Switch system, requiring no tools for fitting


Both bed halves can be moved to a different location on stylishly covered castors

The bed halves can easily be separated and rejoined

Advantages:

- The two halves of the bed can be separated and put back together again effortlessly
- Ideal for phases during which one partner is in need of care, or separate sleeping arrangements are necessary

External dimensions/height adjustment range:

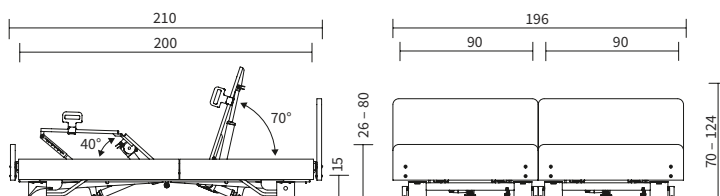



Elegant upholstery or an attractive wood decor, side panels or safety sides – with the Regia partner double bed, couples are spoilt for choice.



Bed halves



- Separating the two halves of the bed facilitates bedside care and makes cleaning the room easier.



Handset

- One for each mattress base

Special features:

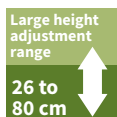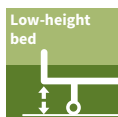

Further information:



- Product page
- This link will take you to the digital Regia partner configuration form.



Product video



Assembly video

Side elements | Regia partner double bed



1 Split safety sides at head and foot ends, telescopic (TSG).



2 Split safety side at head end, telescopic (TSG), and wooden side panels.



3 Split safety side at head end, telescopic (TSG), and fabric/faux leather side panels.



4 Wooden side panels.



5 Fabric/faux leather side panels.

This link will take you to the digital
Regia partner configuration form.



Technical data	Regia partner	
Net weight	270 kg	
External dimensions	196 x 210 cm	
Height adjustment range	26–80 cm	
Safe working load	2 x 225 kg	
Maximum resident weight	195 kg	
24-volt drive system	✓	
Standard features	Regia partner	
Frame colour	Argentum	
Decor	Natural Beech	
Mattress base size	180 x 200 cm	
Metal slatted mattress base (mattress base model)	✓	
Ratchet mechanism, electrical adjustment of lower leg rest for metal slatted mattress base	✓	
Basic model	Regia partner	Material no.
Regia partner care bed 180 x 200 cm	Mattress base and chassis - Two 4-section mattress bases with metal slats - Height adjustment range of 26 - 80 cm - Central braking system and concealed castors for a homelike touch	266800
Side elements	Regia partner	Material no.
① Split safety sides at head and foot ends, telescopic (TSG)	- 2 x TSG 110 cm - 2 x TSG 90 cm - Can be attached at head or foot end as required	SEE_001
② Split safety sides at head end, telescopic (TSG), and wooden side panels	- 2 x TSG at head end 110 cm - 2 x wooden side panels at foot end 90 cm - TSG 110 cm is fully compliant with the standards	SEE_004 SEA_001 SBU_001
③ Split safety side at head end, telescopic (TSG), and wooden side panels	- 2 x TSG at head end 90 cm - 2 x wooden side panels at foot end 110 cm - TSG 90 leaves plenty room for getting in and out of bed	SEE_004 SEA_002 SBU_001
④ Split safety sides at head end, telescopic (TSG), and fabric/faux leather side panels	- 2 x TSG at head end 110 cm - 2 x fabric/faux leather side panels at foot end 90 cm - TSG 110 cm is fully compliant with the standards	SEE_004 SEA_001 SBU_002
⑤ Split safety side at head end, telescopic (TSG), and fabric/faux leather side panels	- 2 x TSG at head end 90 cm - 2 x fabric/faux leather side panels at foot end 110 cm - TSG 90 leaves plenty room for getting in and out of bed	SEE_004 SEA_002 SBU_002
⑥ Wooden side panels	- 2 x wooden side panels 110 cm - 2 x wooden side panels 90 cm - Can be attached at head or foot end as required	SEE_003 SEA_001 SBU_001
⑦ Fabric/faux leather side panels	- 2 x fabric/faux leather side panels 110 cm - 2 x fabric/faux leather side panels 90 cm - Can be attached at head or foot end as required	SEE_003 SEA_001 SBU_002

Accessories | Regia partner double bed



1 Nobla.




2 Nobla mala.




3 Nobla upholstered.



4 Nobla mala upholstered.

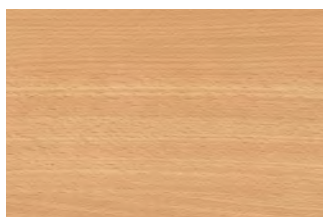

5 Standard decor: Natural Beech.

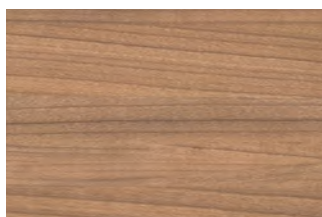


6 Special decor: Havana Cherry.




7 Covering fabric: light green Tiba.




8 Covering fabric: red Tiba.

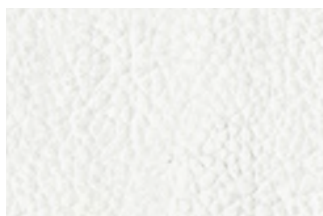

9 Faux leather, white.

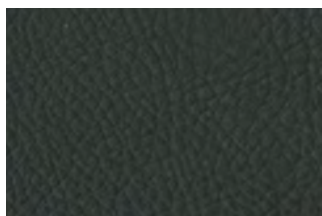

10 Faux leather, Anthracite.

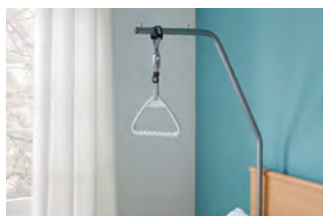


11 Patient lifting pole.



12 Bed extension.



13 14 Comfort mattress base.

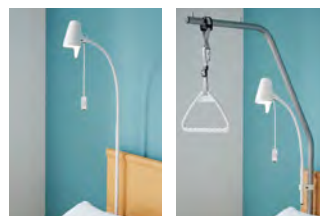

15 Reading lamp for easy insertion.

16 Reading lamp for fixing to the patient lifting pole.



17 Mattress piece for bed extension.



18 19 Foam leather cover for telescopic safety sides (TSG).



20 Sky-fit mattress.



21 Comfort-fit mattress.



22 Basic-fit mattress.

This link will take you to the digital Regia partner configuration form.



Headboards and footboards	Regia partner	Material no.
1 Nobla	- 2 x headboard, Nobla - 2 x footboard, Nobla	KFH_011
2 Nobla mala	- 2 x headboard, Nobla - 2 x low-height footboard, Nobla mala	KFH_013
3 Nobla (upholstered)	- 2 x headboard, upholstered - 2 x footboard, upholstered - Fixed upholstery with non-removable fabric/faux leather	KFH_019
4 Nobla mala (upholstered)	- 2 x headboard, upholstered - 2 x low-height footboard, upholstered - Fixed upholstery with non-removable fabric/faux leather	KFH_020
Wood decors	Regia partner	Material no.
5 Standard decor	Natural Beech R 24010	DHU_027
6 Special decor	Havana Cherry R 42006	DHU_024
Cover materials	Regia partner	Material no.
7 Covering fabric	Light green Tiba 6210	BSH_H006
8 Covering fabric	Red Tiba 3210	BSH_H001
Faux leather	Regia partner	Material no.
9 Faux leather	White 1-1142-197	BSH_K009
10 Faux leather	Anthracite 1-1142-395	BSH_K025
Special equipment and accessories	Regia partner	Material no.
11 Patient lifting pole with triangular grab handle	O	188223
12 Bed extension for metal slatted mattress base 90 cm with telescopic safety sides (TSG)	- Extends the mattress base to 218 cm	LFL_008
13 Comfort mattress base for bed extension	O	279000
14 2 x Comfort mattress base	O	ZLFA003
15 Reading lamp for easy insertion	O	264877
16 Reading lamp for fixing to the patient lifting pole	O	289177
17 Mattress piece for bed extension, 90 cm wide	- Extends the mattress when the bed extension is in use - Dimensions (W x L x H): 90 x 21 x 12 cm - Colour: White - With PU cover	166715
18 Foam leather cover for 90 cm telescopic safety sides (TSG)	O	733670
19 Foam leather cover for 110 cm telescopic safety sides (TSG)	O	733669
Mattresses	Regia partner	Material no.
20 Sky-fit mattress	90 cm x 200 cm x 14 cm	283924
21 Comfort-fit mattress	90 cm x 200 cm x 14 cm	283922
22 Basic-fit mattress	90 cm x 200 cm x 12 cm 90 cm x 220 cm x 12 cm	261477 261484

✓ = Standard feature O= Available as an option




Inovia care bed



Advantages:

- Homelike appearance with opulent head/footboards
- Lowest position is particularly low: 22 cm
- Protects the resident from injuries caused by falls, without the use of liberty-depriving measures
- Optional telescopic safety sides promote the resident's mobility

External dimensions/height adjustment range:

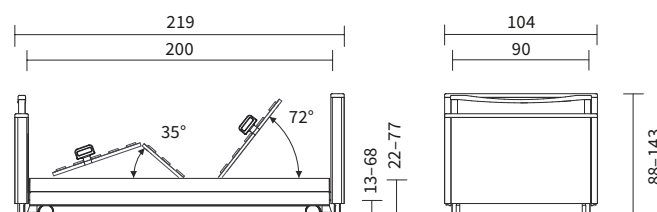

Design details



Handle bar

- Made of solid wood



Castors can be locked in pairs

- Regardless of the mattress base position



Handset

- With selective locking



Sliding backrest



Mattress base height adjustment

- From 22–77 cm
- Adjustment range = 55 cm



Natural Beech (standard)



Havana Cherry

Special features:



Low-height
bed



Large height
adjustment
range

22 to
77 cm



High safe
working
load

225 kg



Werdenfelser
approach

Further information:



- Product page

Accessories and mattresses | Inovia care bed



1 Patient lifting pole.



2 3 Bed extension.



4 Natural Beech decor.



5 Havana Cherry decor.



6 The filler piece can be fitted onto the full-length safety side to make it easier for residents to sit on the edge of the bed.



7 Flexible transmitter holder for the handset.



8 Extension cable for the handset (1.40 m).



9 10 The comfort mattress base comprises 50 free-floating spring elements.



11 12 Sola reading lamp.



13 Mattress piece for bed extension.



14 Foam leather cover for full-length safety sides (DSG).



15 Foam leather cover for telescopic safety sides (TSG).



16 Tray.



17 18 Telescopic safety sides (TSG).



19 Wall deflection roller.



20 Sky-fit care bed mattress.



21 Comfort-fit care bed mattress.



22 Basic-fit care bed mattress.

Technical data		Inovia
Net weight		153/159 kg
External dimensions		104 x 219 cm 114 x 219 cm
Height adjustment range		22 – 77 cm
Safe working load		225 kg
Maximum resident weight		195 kg
24-volt drive system		✓
Standard features		Inovia
Frame colour		Argentum
Decor		Natural Beech
Mattress base size		90 x 200 cm 100 x 200 cm (special size)
Metal slatted mattress base (mattress base model)		✓
Type of handset (selectively lockable/wireless)		Selective
Mechanical CPR release		✓
Castor diameter		100mm
Castor locking		Per axle
Ratchet mechanism, electrical adjustment of lower leg rest		✓
Mechanical CPR release		✓
Design features	Inovia	Material no.
Inovia 90 x 200 cm with full-length safety sides (DSG)		262655
Inovia 100 x 200 cm with full-length safety sides (DSG)		262655
Special equipment and accessories	Inovia	Material no.
1 Patient lifting pole	O	188223
2 Bed extension for metal slatted mattress base 90 and 100 cm with DSG	O	LFL_005
3 Bed extension for metal slatted mattress base 90 and 100 cm with TSG	O	LFL_006
4 Natural Beech decor	✓	DHU_027
5 Havana Cherry decor	O	DHU_024
6 Filler piece (for full-length safety side only)	O	258395
7 Transmitter holder for handset	O	199137
8 Handset extension cable (1.40 m)	O	238047
9 Comfort mattress base	O	245702
10 Comfort mattress base for bed extension	O	257471
11 Reading lamp for easy insertion	O	264877
12 Reading lamp for fixing to the patient lifting pole	O	289177
13 Mattress piece for bed extension 90 cm wide, 12 cm thick	90 cm x 21 cm x 12 cm	166715
14 Foam leather cover, DSG	O	225921
15 Foam leather cover, TSG	O	716556
16 Tray for full-length safety sides (DSG), 90 cm wide	O	267000
17 Telescopic safety sides (TSG) on both sides	O	SHC_002
18 Telescopic safety side (TSG) on one side/DSG on one side	O	SHC_008
19 Wall deflection roller with holder (right or left)	O	Left 215921 Right 221306
Mattresses	Inovia	Material no.
20 Sky-fit mattress	90 cm x 200 cm x 14 cm 100 cm x 200 cm x 14 cm	283924 283925
21 Comfort-fit mattress	90 cm x 200 cm x 14 cm 100 cm x 200 cm x 14 cm	283922 283923
22 Basic-fit mattress	90 cm x 200 cm x 12 cm 90 cm x 220 cm x 12 cm 100 cm x 200 cm x 12 cm 100 cm x 220 cm x 12 cm	261477 261484 261486 261487

Westfalia range



Robert Spencer,
Branch Manager,
medical supplies store

**“Westfalia range –
the timeless beds.**

Many of my customers have always lived a very down-to-earth life and bought their furniture accordingly. The beds from the Westfalia range offer precisely this robust, timeless quality.”



Westfalia IV

The care bed for a more homelike ambiance

- ✓ Safe working load of 185 kg
- ✓ Maximum weight of resident: 155 kg
- ✓ Height adjustment range: 35-80 cm
- ✓ 24-volt drive system
- ✓ No leakage current measurements during expected service life
- ✓ Selectively lockable handset
- ✓ Attractive head and footboard designs
- ✓ Different mattress base sizes to choose from



Westfalia-Klassik

The care bed with special features

- ✓ Safe working load of 200 kg
- ✓ Maximum weight of resident: 170 kg
- ✓ Height adjustment range: 35-80 cm
- ✓ 24-volt drive system
- ✓ No leakage current measurements during expected service life
- ✓ Backrest retraction in accordance with the recommendations of the German Nurses Association (DBfK)
- ✓ Central braking system



Westfalia IV care Bed

Electrically adjustable lower leg rest

185 kg safe working load, 4-section mattress base, electrically adjustable

A selection of homelike wooden surrounds




Robust, high-quality castors

Cost-saving 24-volt drive system

Advantages:

- Combines great comfort, advanced safety and a stylish design
- Height adjustable with a stable, elegant scissor mechanism
- The headboards and footboards are available in various attractive forms

External dimensions/height adjustment range:

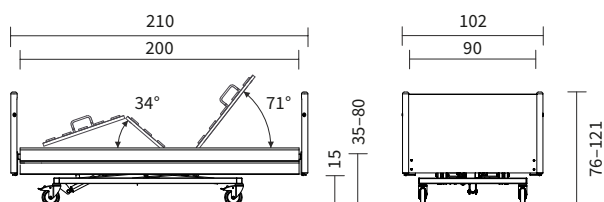

Design details



Handset

- With selective locking function




Castors

- Can be individually braked



Height adjustment range

- 35–80 cm

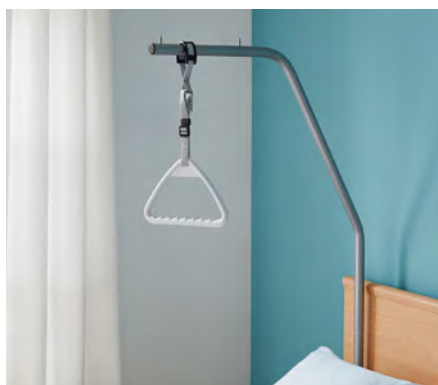

Patient lifting pole



Mattress base

- With sprung wooden slats



Choice of headboards and footboards

- Standard (provided as standard)
- Comodo with round posts (optional)
- Primero with handle bar (optional)

Special features:



Further information:



- Product page



- Assembly video

Accessories and mattresses | Westfalia IV care bed



1 Adaptable safety sides.



2 Standing aid.



3 4 Bed extension including safety side bars (also for retrofitting).



5 Filler piece.



6 Flexible transmitter holder for the handset.



7 Extension cable for the handset (2.40 m).



8 Comodo wooden surround.



9 Primero wooden surround.



10 11 Reading lamps.



12 Mattress piece for bed extension.



13 Foam leather cover.



14 Tray.



15 16 17 Wall deflection roller including mount.



18 Sky-fit care bed mattress.



19 Comfort-fit care bed mattress.




20 Basic-fit care bed mattress.

Technical data	Westfalia IV		
Net weight	112 kg		
External dimensions	110 x 232 cm 110 x 212 cm	100 x 232 cm 100 x 192 cm 100 x 202 cm 100 x 212 cm	90 x 212 cm
Height adjustment range	35–80 cm		
Safe working load	185 kg		
Maximum resident weight	155 kg		
24-volt drive system	✓		
Standard features	Westfalia IV		
Frame colour	Topaz		
Decor	Natural Beech		
Mattress base size	90 x 200 cm		
Sprung wooden slat mattress base (mattress base model)	✓		
Safety side	DSG		
Type of handset (selectively lockable/wireless)	Selective		
Mechanical CPR release	✓		
Castor diameter	125mm		
Castor locking	Single		
Ratchet mechanism, electrical adjustment of lower leg rest	✓		
Patient lifting pole	✓		
Design features	Westfalia IV	Material no.	
Westfalia IV 90 x 200 cm		284050	
Westfalia IV 100 x 200 cm ¹⁾		ZLFB002	
Westfalia IV 90 x 180 cm		ZLFL001	
Westfalia IV 90 x 190 cm		ZLFL010	
Special equipment and accessories	Westfalia IV	Material no.	
1 Adaptable safety sides	Right Left	250138 250139	
2 Standing aid	O	175402	
3 Bed extension to increase mattress base length to 220 cm, including bars for 90 cm width	O	291754	
4 Bed extension to increase mattress base length to 220 cm, including bars for 100 cm width	O	301121	
5 Filler piece (for full-length safety side only)	O	258395	
6 Transmitter holder for handset	O	199137	
7 Handset extension cable (2.40 m)	O	178986	
8 Comodo wooden surround with round posts	O	ZMOK015	
9 Primero wooden surround	O	ZMOK017	
10 Reading lamp for easy insertion	O	264877	
11 Reading lamp for fixing to the patient lifting pole	O	289177	
12 Mattress piece for bed extension 90 cm wide, 12 cm thick	90 cm x 21 cm x 12 cm	166715	
13 Foam leather cover, DSG	O	225921	
14 Tray for full-length safety sides (DSG), 90 cm wide	O	267000	
15 2 wall deflection rollers, horizontal action, foot end	O	ZWAN003	
16 2 wall deflection rollers, horizontal action, head end	O	ZWAN002	
17 4 wall deflection rollers, horizontal action	O	ZWAN004	
Mattresses	Westfalia IV	Material no.	
18 Sky-fit mattress	90 cm x 200 cm x 14 cm 100 cm x 200 cm x 14 cm	283924 283925	
19 Comfort-fit mattress	90 cm x 200 cm x 14 cm 100 cm x 200 cm x 14 cm	283922 283923	
20 Basic-fit mattress	90 cm x 200 cm x 12 cm 100 cm x 200 cm x 12 cm 90 cm x 180 cm x 12 cm 90 cm x 190 cm x 12 cm	261477 261486 261482 261483	




Westfalia-Klassik care bed

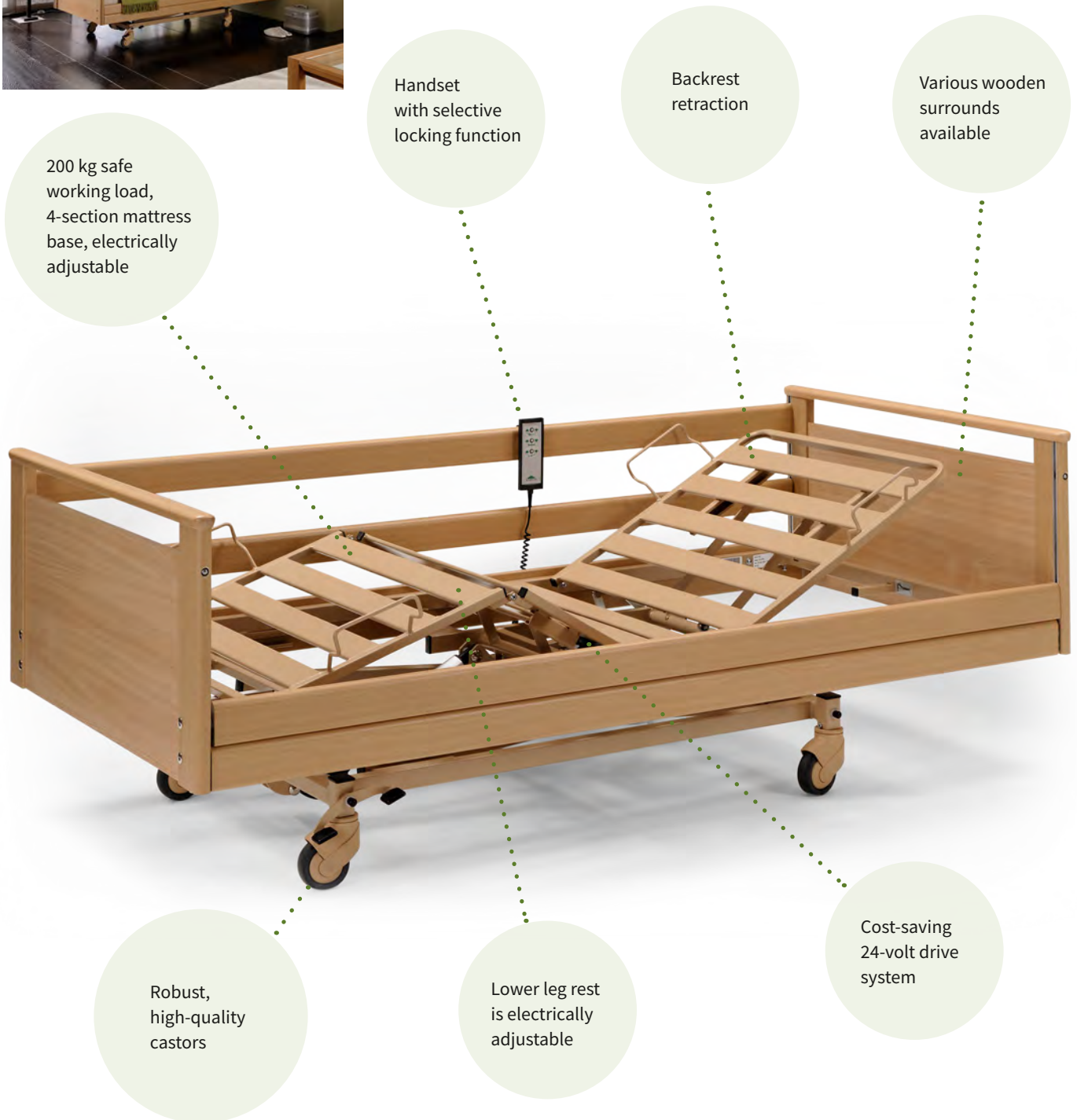

Advantages:

- Well-being and individuality
- Sophisticated technical features

External dimensions/height adjustment range:

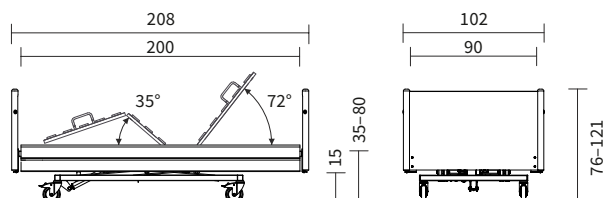

Design details



Central castor locking

- Possible whatever the mattress base position



Electronic power supply unit

- SwitchMode, integrated immediately downstream of the mains plug



Mattress base height adjustable

- From 35-80 cm
- Adjustment range = 45 cm

Special features:



Further information:



- Product page

Accessories | Westfalia-Klassik care bed



1 Bed extension with metal slats, safety sides and side panels.



2 Filler piece to place over the full-length safety side.



3 Flexible transmitter holder for the handset.



4 Extension cable for the handset (2.40 m).



5 6 Wooden surrounds: Standard (standard feature), Comodo with round posts (optional), Primero with handle bar (optional).



7 8 Reading lamps.



9 Mattress piece for bed extension.



10 Foam leather cover.



11 Tray.



12 Wall deflection roller.



13 Sky-fit care bed mattress.



14 Comfort-fit care bed mattress.



15 Basic-fit care bed mattress.

Technical data	Westfalia-Klassik	
Net weight	132 kg	
External dimensions with standard wooden surround	102 x 208 cm	
Height adjustment range	35–80 cm	
Safe working load	200 kg	
Maximum resident weight	170 kg	
24-volt drive system	✓	
Standard features	Westfalia-Klassik	
Frame colour	Topaz	
Decor	Natural Beech	
Mattress base size	90 x 200 cm	
Safety side	DSG	
Type of handset (selectively lockable/wireless)	Selective	
Mechanical CPR release	✓	
Castor diameter	100mm	
Castor locking	Central	
Ratchet mechanism, electrical adjustment of lower leg rest	✓	
Design features	Westfalia-Klassik	Material no.
Westfalia-Klassik 90x200 cm with standard wooden surround		285870
Special equipment and accessories	Westfalia-Klassik	Material no.
1 Bed extension	O	291763
2 Filler piece (for full-length safety side only)	O	258395
3 Transmitter holder for handset	O	199137
4 Handset extension cable (2.40 m)	O	178986
5 Comodo wooden surround with round posts	O	ZMOK015
6 Primero wooden surround	O	ZMOK017
7 Reading lamp for easy insertion	O	264877
8 Reading lamp for fixing to the patient lifting pole	O	289177
9 Mattress piece for bed extension 90 cm wide, 12 cm thick	90 cm x 21 cm x 12 cm	166715
10 Foam leather cover, DSG	O	225921
11 Tray for full-length safety sides (DSG)	O	267000
12 Wall deflection roller Ø 100 mm	O	Right: 176511 Left: 176490
Mattresses	Westfalia-Klassik	Material no.
13 Sky-fit mattress	90 cm x 200 cm x 14 cm	283924
14 Comfort-fit mattress	90 cm x 200 cm x 14 cm	283922
15 Basic-fit mattress	90 cm x 200 cm x 12 cm	261477

Lenus



Maggy Smith,
occupational therapist

**“Lenus – the bed for
floor-level care at home.**

Low-height beds such as the Lenus offer an easy way of dramatically increasing the level of fall protection for users and offer good protection to dementia sufferers in particular.”




Lenus

The low-height bed for optimal fall prevention

- ✓ Large height adjustment range of 15 – 80 cm
- ✓ Automatic intermediate stops at 25 cm and at the standing up and transfer height of 38 cm
- ✓ Integrated 20-cm bed extension
- ✓ High safe working load of 225 kg
- ✓ Tool-free installation of full-length safety side with Easy Click
- ✓ Pinch protection at the lowest height through an acoustic signal and reduced-speed height adjustment




Lenus low-height bed

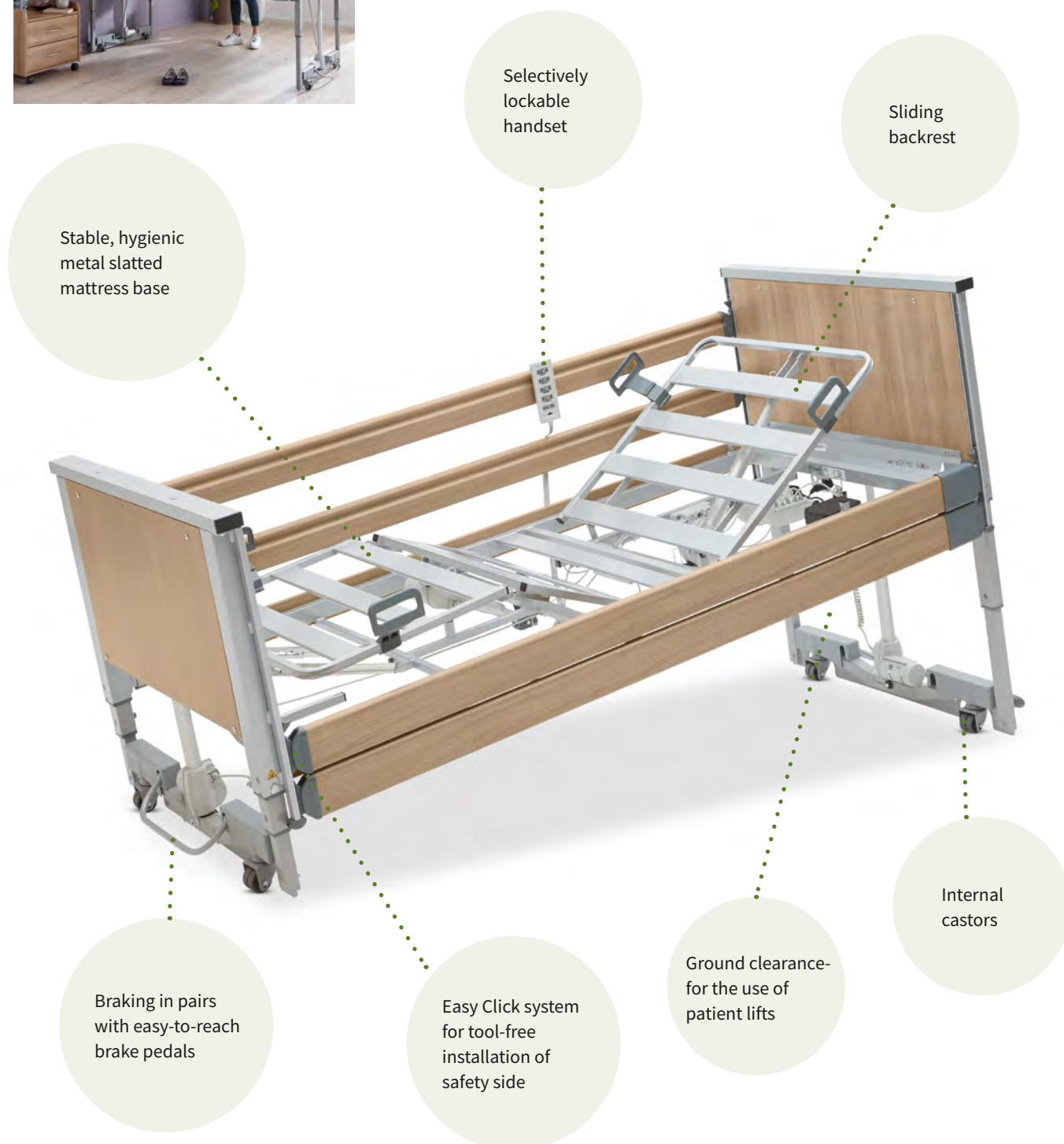

Advantages:

- The floor-level low-height bed protects particularly fall-prone users without restricting their mobility
- With its large height adjustment range from 15 to 80 cm, it meets all the requirements of a normal care bed and ensures back-friendly working
- It is particularly suitable for users with severe cognitive impairments, such as those caused by advanced dementia

External dimensions/height adjustment range:

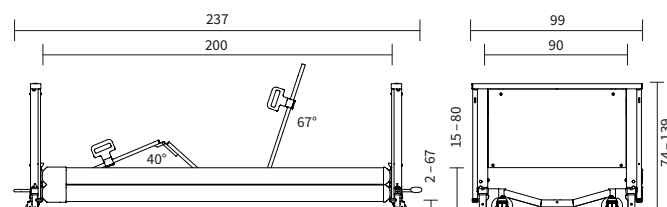

Design details



Ergonomic sitting position.



Ideal height for care-giving.




Tool-free removal/installation of safety sides.



Brake pedal accessible from the side.

Special features:

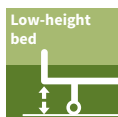

Further information:



Product page




Assembly video

Accessories | Lenus low-height bed



1 External battery

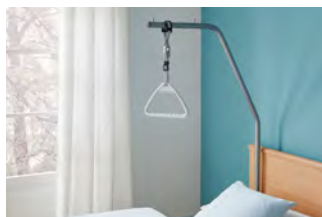

2 Patient lifting pole.



3 Flexible transmitter holder for the handset.



4 The filler piece can be fitted onto the full-length safety side to make it easier for residents to sit on the edge of the bed.




5 Extension cable for the handset (2.0 m).



6 7 Homelike wooden surrounds for the head and footboards.



8 9 Comfort mattress base.

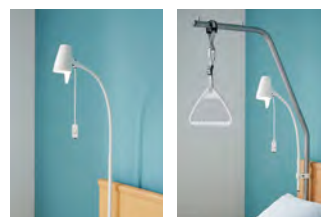

10 Reading lamp for easy insertion.
11 Reading lamp for fixing to the patient lifting pole.



12 Mattress piece for bed extension.



13 Foam leather cover.



14 Tray.



15 Elegant Softcovers for head/footboards and safety sides – available in six different colours.



16 Trendelenburg handset.



17 The bars of the safety sides can be transported and stored even more easily with the carry bag.



18 Under bed light.



Vanilla

Bordeaux



Petrol

Dark blue



Lavender

Brown



19 The optional extension set allows ideal use of the integrated bed extension.



20 Sky-fit care bed mattress.




21 Comfort-fit care bed mattress.



22 Basic-fit care bed mattress.

Technical data	Lenus	
Net weight	123 kg	
External dimensions	99 x 237 cm	
Height adjustment range	15–80 cm	
Safe working load	225 kg	
Maximum resident weight	195 kg	
24-volt drive system	✓	
Standard features	Lenus	
Frame colour	Argentum	
Decor	Natural Beech	
Mattress base size	90 x 200 cm	
Metal slatted mattress base (mattress base model)	✓	
Safety side	DSG	
Type of handset (selectively lockable/wireless)	Selective	
Mechanical CPR release	✓	
Castor diameter	50mm	
Castor locking	Per axle	
Ratchet mechanism, electrical adjustment of lower leg rest	✓	
Design features	Lenus	Material no.
Lenus 90 x 200 cm Natural Beech decor (230-volt drive system / Type G plug)		287378
Lenus 90 x 200 cm Havana Cherry decor (230-volt drive system / Type G plug)		287381
Lenus 90 x 200 cm Natural Beech decor (24-volt drive system)		287382
Lenus 90 x 200 cm Havana Cherry decor (24-volt drive system)		287383
Special equipment and accessories	Lenus	Material no.
1 Rechargeable battery	O	288376
2 Patient lifting pole	O	188223
3 Flexible transmitter holder for handset	O	199137
4 Filler piece for the long side of the bed frame	O	258395
5 Handset extension cable (2 metres)	O	238050
6 Wooden surrounds for the head and footboards, Natural Beech decor	O	293721
7 Wooden surrounds for the head and footboards, Havana Cherry decor	O	293727
8 Comfort mattress base 200 cm	O	247814
9 Comfort mattress base 220 cm (with bed extension)	O	279000
10 Reading lamp for easy insertion	O	264877
11 Reading lamp for fixing to the patient lifting pole	O	289177
12 Mattress piece for bed extension	O	209240
13 Foam leather cover	O	225921
14 Tray	O	267000
15 Softcover for safety side	Vanilla	259549
	Bordeaux	259555
	Petrol	259557
	Dark blue	259556
	Brown	259551
16 Trendelenburg handset (for use in nursing homes only)	O	277989
17 Carry bag for safety side bars	O	273831
18 Under bed light	O	277987
19 Extension set with extension piece for mattress base, bars and panel infill pieces	O	290195
Mattresses	Lenus	Material no.
20 Sky-fit mattress	90 cm x 200 cm x 14 cm	283924
21 Comfort-fit mattress	90 cm x 200 cm x 14 cm	283922
22 Basic-fit mattress	90 cm x 200 cm x 12 cm	261477

Heavy duty beds

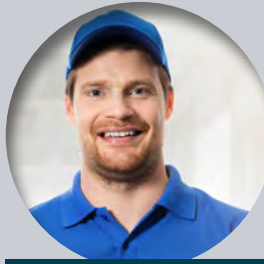

Dave Johnson,
rehabilitation clinic
technician

“Allura II – the heavy-duty bed that makes my work easy.

For us technicians, assembling large, sturdy, heavy-duty beds is often challenging. This is why I am particularly pleased with the Easy Click system of the Allura II range. Now I can attach the safety sides in a matter of seconds.”

Gigant

The heavy-duty care bed



- ✓ Safe working load of 350 kg
- ✓ Maximum weight of resident: 280 kg
- ✓ Height adjustment range: 40-80 cm
- ✓ Reverse-Trendelenburg position
- ✓ Optional combinable safety side
- ✓ Headboards and footboard with elegant wooden surround

Allura II

Innovative care beds



- ✓ Safe working load of 250 kg
- ✓ Maximum weight of resident: 210 kg
- ✓ Height adjustment range: 30-80 cm
- ✓ 24-volt drive system
- ✓ No leakage current measurements during expected service life
- ✓ Reverse-Trendelenburg position
- ✓ Selectively lockable handset
- ✓ Mattress base widths of 100 and 120 cm
- ✓ Tool-free safety side installation with Easy Click system




Gigant heavy-duty bed



Advantages:

- Supports a body weight of up to 280 kg, making it suitable for extremely obese persons
- Plenty of space and support for independent living
- Relieves the strain on carers

External dimensions/height adjustment range:

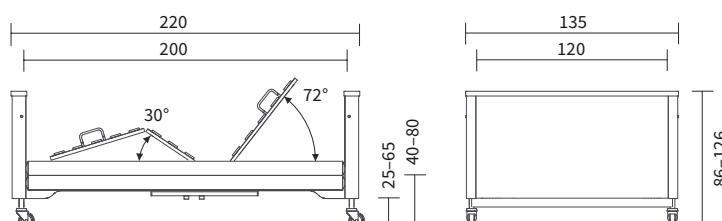


Design details



Adapter sleeve for mobilisation support (left) and locking box (right)



Emergency release of the backrest

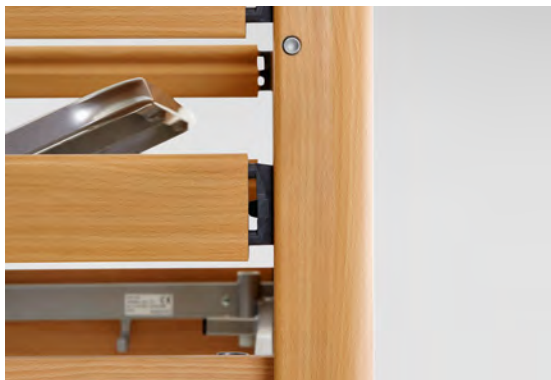

Extremely sturdy safety side construction



Plastic cladding around chassis
• Ensures a homelike feel

Special features:



Further information:



- Product page

Accessories | Gigant heavy-duty bed



1 Clamp-on extra-high safety side rail.



2 Bed extension including longer safety sides and side panels.



3 Filler piece to place over the full-length safety side.



4 5 6 Combinable safety side.



7 8 Reading lamps.



9 Mattress piece for bed extension.



10 11 Foam leather cover.




12 Cloud-fit heavy-duty mattress.

Technical data	Gigant	
Net weight	159 kg	
External dimensions with standard wooden surround	135 x 222 cm	
Height adjustment range	40–80 cm	
Safe working load	350 kg	
Maximum resident weight	280 kg	
Standard features	Gigant	
Frame colour	Argentum	
Decor	Natural Beech	
Mattress base size	120 x 200 cm	
Safety side	DSG	
Type of handset (selectively lockable/wireless)	Selective	
Mechanical CPR release	✓	
Castor diameter	100mm	
Castor locking	Single	
Ratchet mechanism, electrical adjustment of lower leg rest	✓	
Design features	Gigant	Material no.
Gigant 120 x 200 cm		286960
Special equipment and accessories	Gigant	Material no.
① Clamp-on extra-high safety side rail	O	164977
② Bed extension for metal slatted mattress base (with DSG)	O	291767
③ Filler piece (for full-length safety side only)	O	258395
④ Combinable safety side on both sides	O	ZSGN044
⑤ Combinable safety side on left side	O	ZSGN045
⑥ Combinable safety side on right side	O	ZSGN046
⑦ Reading lamp for easy insertion	O	264877
⑧ Reading lamp for fixing to the patient lifting pole	O	289177
⑨ Mattress piece for bed extension 120 cm wide, 19 cm thick	120 cm x 21 cm x 16 cm 120 cm x 21 cm x 19 cm	241942m 712167
⑩ Foam leather cover DSG, one side	O	225921
⑪ Foam leather cover KSG, one side	O	2 x 183008 + 1 x 712961
Mattresses	Gigant	Material no.
⑫ Cloud-fit mattress	120 cm x 200 cm x 16 cm	300406




Care beds from the Allura II range

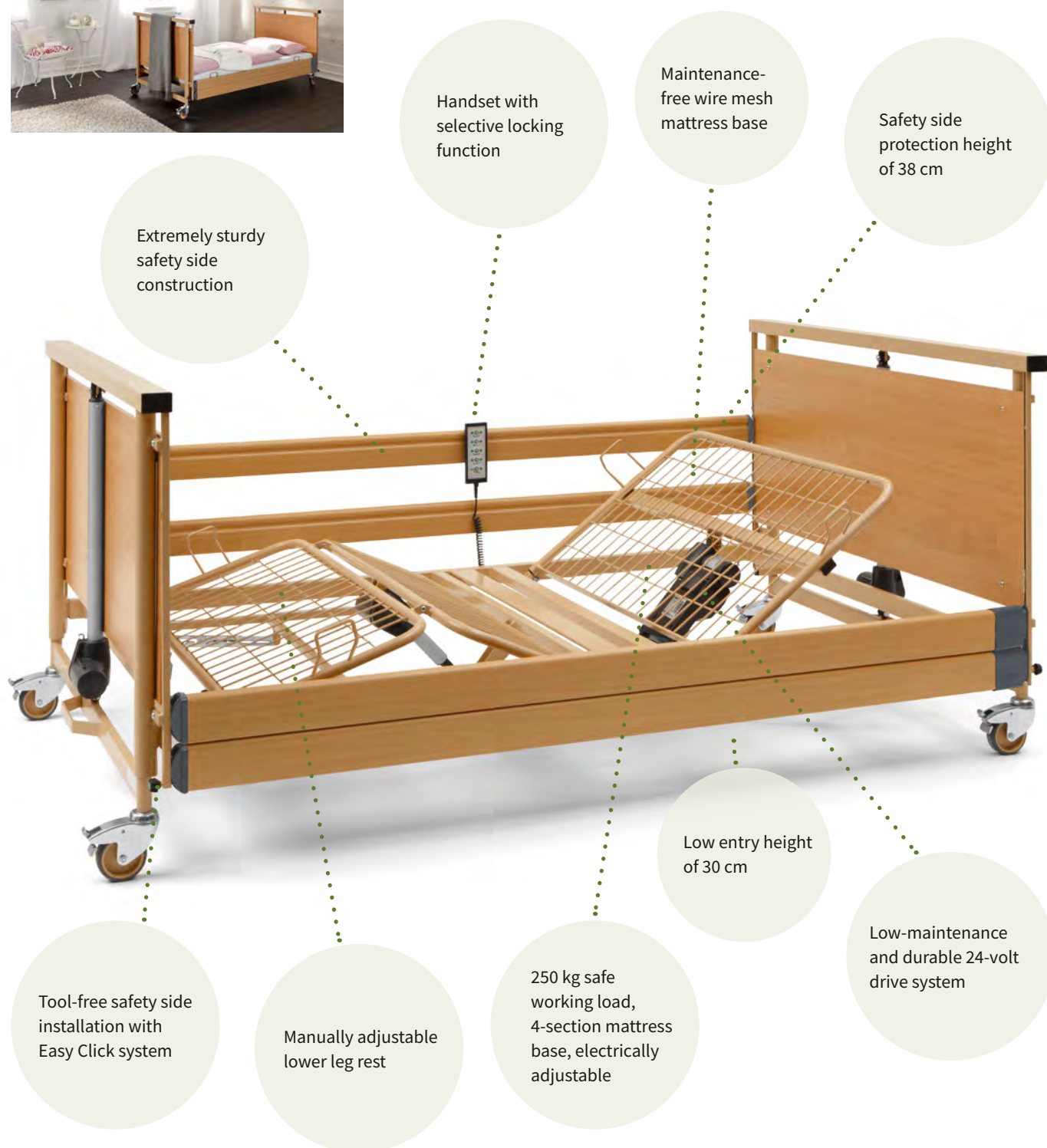

Advantages:

- High load capacity
- Choice of mattress base widths
- Large height adjustment range
- High level of comfort and safety even for heavy persons
- Relieves the strain on carers

External dimensions/height adjustment range:

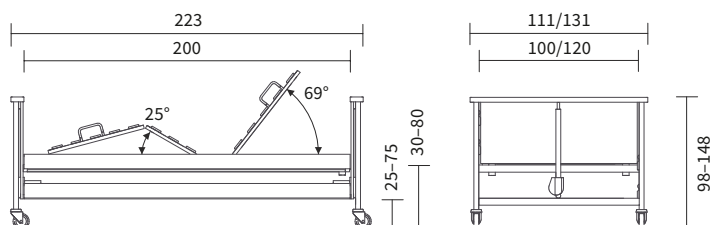

Design details



Reverse-Trendelenburg position



Adjustable from 30 to 80 cm



Electronic power supply unit

- SwitchMode, integrated immediately downstream of the mains plug



Handset

- With selective locking function



Compact delivery and storage on transportation/storage aid (similar to illustration).



Tool-free removal/installation of safety sides

- With Easy Click

Special features:



Further information:




- Product page

Allura II accessories



1 Standing aid.

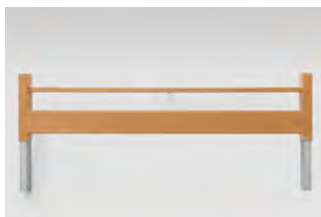

2 3 Bed extension (also for retrofitting).



4 Filler piece to place over the full-length safety side.



5 Extension cable for the handset (1.40 m).



6 7 Wooden panels to cover the chassis (also for retrofitting).



8 9 Reading lamps.



10 Mattress piece for bed extension.



11 Foam leather cover.



12 Wall deflection roller including mount.



A new safety-side height-extension rail will be available soon.



13 Sky-fit care bed mattress.



14 Comfort-fit care bed mattress.



15 Basic-fit care bed mattress.



16 Cloud-fit heavy-duty mattress.

Technical data	Allura II	
Net weight	120 kg	
External dimensions	111 x 224 cm 131 x 224 cm	
Height adjustment range	30–80 cm	
Safe working load	250 kg	
Maximum resident weight	210 kg	
24-volt drive system	✓	
Standard features	Allura II	
Frame colour	Topaz	
Decor	Natural Beech	
Mattress base size	100 x 200 cm 120 x 200 cm	
Safety side	DSG	
Type of handset (selectively lockable/wireless)	Selective	
Mechanical CPR release	✓	
Castor diameter	100mm	
Castor locking	Single	
Ratchet mechanism, electrical adjustment of lower leg rest	✓	
Design features	Allura II	Material no.
Allura II 100 x 200 cm		289450
Allura II 120 x 200 cm		289451
Special equipment and accessories	Allura II	Material no.
① Standing aid	O	244029
② Bed extension for metal slatted mattress base 100 cm	O	291745
③ Bed extension for metal slatted mattress base 120 cm	O	291748
④ Filler piece (for full-length safety side only)	O	258395
⑤ Handset extension cable (1.4 m)	O	238047
⑥ Wooden surround (Allura II 100 / 1 piece)	O	291008
⑦ Wooden surround (Allura II 120 / 1 piece)	O	291006
⑧ Reading lamp for easy insertion	O	264877
⑨ Reading lamp for fixing to the patient lifting pole	O	289177
⑩ Mattress piece for bed extension 120 cm wide, 16 cm thick	120 cm x 21 cm x 16 cm 120 cm x 21 cm x 19 cm	241942 712167
⑪ Foam leather cover, DSG	O	225921
⑫ Wall deflection roller, 1 piece, including mount	O	206751
Mattresses	Allura II	Material no.
⑬ Sky-fit mattress	100 cm x 200 cm x 14 cm	283925
⑭ Comfort-fit mattress	100 cm x 200 cm x 14 cm	283923
⑮ Basic-fit mattress	100 cm x 200 cm x 12 cm 100 cm x 220 cm x 12 cm	261486 261487
⑯ Cloud-fit mattress	100 cm x 200 cm x 16 cm 120 cm x 200 cm x 16 cm	300900 300406

Bedside cabinets and side tables



Robert Spencer,
Branch Manager,
medical supplies store

“Bedside cabinets – attractive options for our customers.

To go with a comfortable care bed you also need a suitable bedside cabinet on which to place drinks or care utensils. It’s great that we can offer our customers additional options in the form of Burmeier bedside cabinets, to increase their quality of life.”



James Borrow,
medical product consultant

“Bedside cabinets – an underappreciated companion in everyday life.

When I advise my clients, they often feel that a bedside cabinet isn’t really necessary – any old side table will do, they think. After just a few days of care at home, however, many realise that this tailor-made companion is indispensable.”

Bedside cabinets and side tables



Practical and functional

When people requiring care are not able to leave their bed very often, it is all the more important that everyday things are within reach. Our feelgood bedside cabinets offer storage space and surfaces for placing care utensils, books and magazines, phones or a radio. With their help, it is much easier for residents to take part in everyday life. Fold-down, height-adjustable overbed tables are also ideal for eating meals in bed. Thanks to their castors, the bedside cabinets are easy to use anywhere.

Bedside cabinets and side tables



Item number: 193753

Cherusker bedside cabinet

With height-adjustable tilting overbed table, plastic-coated body, Beech decor.

Technical data:

Width: 52cm
Height: 87cm
Overall depth: 50cm
Overall width (overbed table horizontal): 114cm
Overbed table height adjusts from 82 to 112 cm
Double castors Ø 50 mm, 2 locking castors



Item number: 236959

Arminius bedside cabinet

With height-adjustable tilting overbed table, plastic-coated body, Beech decor.

Technical data:

Width: 54.5cm
Height: 89cm
Overall depth: 45cm
Overall width (overbed table horizontal): 125cm
Overbed table height adjustment from 78 to 112 cm
Double castors Ø 50 mm, 2 locking castors





Item number: 287111

Hermann bedside cabinet with overbed table

With fold-down overbed table, plastic-coated body, beech decor.

Technical data:

Width: 52.5cm

Height: 89cm

Overall depth: 45.5cm

Overall width (overbed table horizontal): 114cm

Double castors Ø 50 mm, 2 locking castors



Item number: 287112

Hermann bedside cabinet without overbed table

With fold-down overbed table, plastic-coated body, beech decor.

Technical data:

Width: 44.5cm

Height: 89cm

Overall depth: 45cm

Double castors Ø 50 mm, 2 locking castors



Item number: 287103

Varus bedside cabinet

Plastic-coated body, no overbed table,
Available in four different decors: Natural Beech, Royal
Maple, Havana Cherry, Bella Noce Choco.

Technical data:

Width: 54cm

Height: 57cm

Overall depth: 45cm

Double castors Ø 50 mm, 2 drawers

Further information:



Product page



Item number: 218931

Duo overbed table and side table

With height-adjustable, tilting overbed table and a fixed table surface, Beech decor.

Technical data:

Width: 81cm

Height: 81cm

Overall depth: 42cm

Overbed table height adjusts from 80 to 113 cm

Double castors Ø 50 mm, 2 castors are lockable



Item number: 218932

Solo overbed table and side table

With height-adjustable tilting overbed table, Beech decor.

Technical data:

Width: 60cm

Height: 81cm

Overall depth: 42cm

Overbed table height adjusts from 80 to 113 cm

Double castors Ø 50 mm, 2 castors are lockable



Further information:



Product page

*Mattresses, transport chair
and furniture*



Robert Spencer,
Branch Manager,
medical supplies store

“Mattresses – important helpers in care at home.

In home care, it isn't wise to settle for standard mattresses. For every Burmeier bed, we offer a perfectly matching mattress from the same manufacturer. This is the only way we can be sure that our customers have the comfort and protection they need.”



Jules Hammond,
customer

“I feel good on my Burmeier mattress.

With my flexible Sky-fit mattress from Burmeier, I can really enjoy the sitting position and all the other adjustment options that my bed offers. The reinforced edge zones also make it easier for me to get out of bed without help.”

Mattresses



Well-being through comfort

- ✓ Pressure ulcer protection
- ✓ Reinforced edge zones
- ✓ Optimal lying comfort
- ✓ Pressure distribution protects against bedsores
- ✓ Reversible mattresses with one firm and one soft side
- ✓ Removable reversible cover with a soft comfort side and a robust hygienic side

Ravello



The transport chair for extra mobility

- ✓ Safe working load of 135 kg (dynamic), 200 kg (static)
- ✓ The lowerable armrests make it easy to get into the chair from the side
- ✓ The backrest can be lowered through 43° to a reclining position
- ✓ Ravello-sano: retractable footrest
- ✓ Ravello-curo: permanently fixed footrest moves with the lower leg rest during adjustments
- ✓ Ravello sano-s: 4 steerable castors for easy manoeuvring
- ✓ Ravello plus-s: height-adjustable seat

Wardrobes and cabinets



Flexible and functional

- ✓ Adrano furniture system with wardrobes and cupboards
- ✓ Options for creating personalised designs for bedrooms and living rooms
- ✓ Clear lines and a homelike wood decor
- ✓ Long service life thanks to high-quality materials
- ✓ Back panels with a thickness of 8 mm
- ✓ Wipe-clean base panel protects against moisture
- ✓ Underside of base panel is protected with ABS edging

Chairs



Sitting up and taking part in everyday life

- ✓ Suitable for residents' rooms as well as for communal and waiting areas in residences
- ✓ Ergonomic seat heights and armrests to offer secure support
- ✓ Users are able to take part in daily activities at eye level and maintain their mobility.
- ✓ Stacking chairs can be quickly and easily stacked to save space
- ✓ This is practical for floor cleaning as well as when reorganising the room for joint events

Mattresses



Beds and
mattresses from
a single source

Advantages:

- All mattress versions ensure a healthy, dry microclimate
- They adapt flexibly to the shape of the mattress base
- Pressure points are reduced
- The bed and mattress can be selected as a perfectly matching unit
- Pressure ulcer protection
- Long-term adaptation to the body shape of the user
- Uniform pressure relief

Further information:



Product page

Dimensions

Model	Dimensions (W x L x H)	Item no.
1 Sky-fit care bed mattress	90 cm x 200 cm x 14 cm	283924
	100 cm x 200 cm x 14 cm	283925
2 Comfort-fit care bed mattress	90 cm x 200 cm x 14 cm	283922
	100 cm x 200 cm x 14 cm	283923
3 Basic-fit care bed mattress	80 cm x 180 cm x 12 cm	261478
	80 cm x 190 cm x 12 cm	261479
	80 cm x 200 cm x 12 cm	261481
	90 cm x 180 cm x 12 cm	261482
	90 cm x 190 cm x 12 cm	261483
	90 cm x 200 cm x 12 cm	261477
	90 cm x 220 cm x 12 cm	261484
	100 cm x 190 cm x 12 cm	261485
	100 cm x 200 cm x 12 cm	261486
	100 cm x 220 cm x 12 cm	261487
	140 cm x 200 cm x 16 cm	261489
4 Cloud-fit heavy-duty mattress	100 cm x 200 cm x 16 cm	300900
	120 cm x 200 cm x 16 cm	300406
	140 cm x 200 cm x 16 cm	300407

Mattresses for a new sleep sensation



Sky-fit care bed mattress (Art. no. 283924)

The premium model to satisfy every requirement

- The high-quality dual-layer mattress for Burmeier beds has a 2-cm-thick Sky-therm® layer and a 12-cm base layer made of cold foam
- Reinforced edge zones
- Adapts perfectly to the mattress base and follows every bed adjustment
- Removable, washable and reversible cover; one side is PU and the other polyester and viscose
- Compression hardness 5.0 kPa (edge zones), 4.0 kPa (base layer), 2.5 kPa (top layer), density 50 kg/m³ (base), 40 kg/m³ (top)



Comfort-fit care bed mattress (Art. no. 300900)

The high-quality mid-range mattress for healthy sleep

- The comfortable mattress with a core made of elastic polyurethane cold foam (14 cm) for recuperative sleep
- Reinforced edge zones
- Adapts perfectly to the mattress base and follows every bed adjustment
- Removable, washable and reversible cover; one side is PU and the other polyester and viscose
- Compression hardness 5.0 kPa (edge zones), 4.0 kPa (base layer), density 40 kg/m³



Basic-fit care bed mattress (Art. no. 261477)

The basic entry-level model for everyday care

- The modern care bed mattress with a polyurethane core (12 cm)
- Washable PU cover, white, removable, with zip fastening
- Density: 40 kg/m³, compression hardness: 4.5 kPa

Cloud-fit heavy-duty mattress (Art. no. 283924)

The versatile heavy-duty mattress

- Particularly suitable for very heavy users with a body weight of 140 to 240 kg
- 3-layer mattress with a top layer made of comfortable Sky-therm® foam, and two resilient cold-foam layers
- Helps prevent pressure ulcers thanks to punched holes in the Sky-therm® foam; excellent point elasticity ensures pressure relief at sensitive points
- Optimal adaptation to the adjustments of the mattress base, thanks to incisions in the bottom cold-foam layer
- Removable reversible cover, one side polyurethane (PU), the other side polyester and viscose, which can be washed in a standard washing machine
- Mattress height of approx. 16 cm, which ensures a standard-compliant protective height of the safety sides on Burmeier care beds



1 Sky-fit mattress without cover in the Regia care bed. Flexible Sky-therm® layer on top, and a base layer with incisions on the bottom.



2 Comfort-fit mattress without cover in the Inovia care bed. The side with the incisions can be turned to the top or bottom.



3 Basic-fit mattress with cover, in the Dali care bed.



4 Cloud-fit mattress without cover in the Gigant heavy-duty bed. A top layer of viscoelastic Sky-therm®, and two resilient layers of cold foam underneath.


Ravello transport chair



Advantages:

- Ensures safe transport of people in need of care
- Its flexible design makes it easy for carers to transfer the user to the chair from an armchair or from the side of the bed.
- The cushioned, ergonomically shaped seat also makes the Ravello comfortable to sit in for longer periods.
- Easy-care, washable seat covers

External dimensions/height adjustment range:

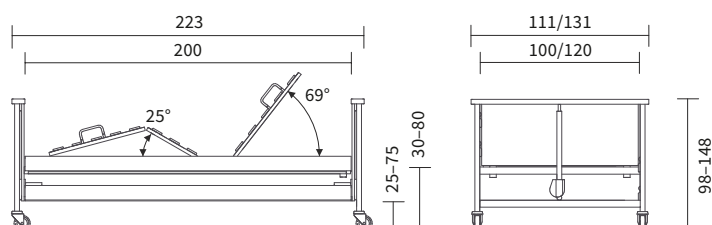

Design details



Ravello sano

- The footrest can be completely retracted.



Ravello curo

- The length-adjustable, fixed position footrest that tilts to follow backrest adjustments



Armrest height adjustment

- In 6 stages



Ravello sano-s

- For better manoeuvrability

Further information:



[Product page](#)

Accessories for Ravello



1 2 Leg supports.



3 Infusion stand.



4 Headrest with lateral support.



5 Side supports.



6 Attachable table.



7 Urine bag holder.

Technical data		Ravello	
Net weight		curo, sano, sano-s: 38 kg plus-s: 58 kg	
Dimensions		curo, sano, sano-s: W x H x D 68 x 126 x 103 cm incl. footrests plus-s: W x H x D 73 x 132–161 x 109 cm incl. footrests	
Backrest		Height 81 cm, reclines to 43°	
Footrest		26 x 24 cm, with 8 cm ground clearance	
Castors		Front castors: 20 cm, fixed Rear castors: 15 cm, swivel type, with paired locking	
S version castors:		15 cm, swivel type, with paired locking	
Seat		PU foam with wooden core	
Backrest		PU foam with steel core	
Covering fabric		Faux leather	
Design features		Ravello	Material no.
Ravello curo			264299
Ravello sano			264297
Ravello sano-s			264298
Ravello plus-s			264300
Special equipment and accessories		Ravello	Material no.
① Leg support, left		O	116343
② Leg support, right		O	116342
③ Infusion stand		O	1x 260269 1x 114352
④ Neck support (black)		O	116344
⑤ 1 pair of side supports (azure blue)		O	189619
⑥ Tray, attachable table		O	195900
⑦ Urine bag holder		O	116340

Wardrobes/linen cupboards, sideboards



Adrano


offers a variety of different options for keeping valuables safe. All wardrobes and cupboards can be fitted with optional safes.

These lockable compartments can also be built in as permanent fixtures.

Mix and match

Take advantage of the wide range of furniture units available to create your very own wardrobe and cabinet combination.


Examples of combinations

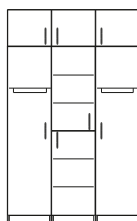

Combination of
278693 & 278698

Technical data:


Width: 150 cm
Height: 244 cm
Overall depth: 60 cm

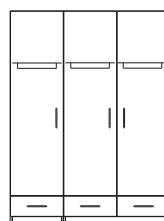

Combination of
278692 & 278681

Technical data:


Width: 150 cm
Height: 202 cm
Overall depth: 60 cm
with 3 base drawers

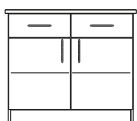

Material no.: 278711*

Technical data:


Width: 100 cm
Height: 90 cm
Depth: 52 cm
with 2 drawers

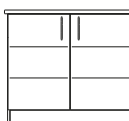

Material no.: 278712*

Technical data:


Width: 100 cm
Height: 90 cm
Depth: 52 cm

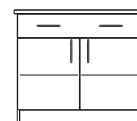

Material no.: 278713*

Technical data:


Width: 100 cm
Height: 90 cm
Depth: 52 cm
with 1 drawer

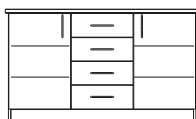

Material no.: 278714*

Technical data:

Width: 150 cm
Height: 90 cm
Depth: 52 cm
with 4 drawers

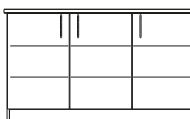

Material no.: 278715*

Technical data:

Width: 150 cm
Height: 90 cm
Depth: 52 cm

Further information:




Product page


*If sideboards or cupboards are combined, the adjoining top surface of the sideboard ends flush with the sideboard body.

Wardrobes/linen cupboards, top units, accessories


One-person wardrobes/linen cupboards*

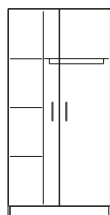


Material no.: 278682
Technical data:
 Width: 100 cm
 Height: 202 cm
 Depth: 60 cm

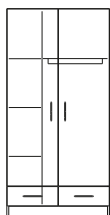


Material no.: 278683
Technical data:
 Width: 100 cm
 Height: 202 cm
 Depth: 60 cm
 with 2 base drawers

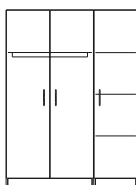


Material no.: 278686
Technical data:
 Width: 150 cm
 Height: 202 cm
 Depth: 60 cm

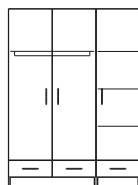


Material no.: 278687
Technical data:
 Width: 150 cm
 Height: 202 cm
 Depth: 60 cm
 with 3 base drawers

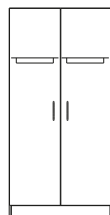


Material no.: 278691
Technical data:
 Width: 100 cm
 Height: 202 cm
 Depth: 60 cm

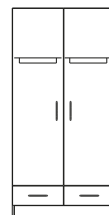

Material no.: 278692
Technical data:
 Width: 100 cm
 Height: 202 cm
 Depth: 60 cm
 with 2 base drawers

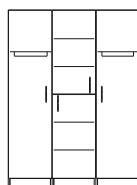

Material no.: 278693
Technical data:
 Width: 150 cm
 Height: 202 cm
 Depth: 60 cm

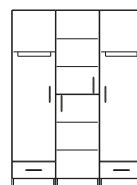

Material no.: 278694
Technical data:
 Width: 150 cm
 Height: 202 cm
 Depth: 60 cm
 with 2 base drawers

Extension elements



Material no.: 278680
Technical data:
 Width: 50 cm
 Height: 202 cm
 Depth: 60 cm



Material no.: 278681
Technical data:
 Width: 50 cm
 Height: 202 cm
 Depth: 60 cm
 with 1 base drawer

Tops




Material no.: 278697
Technical data:
 Width: 100 cm
 Height: 42 cm
 Depth: 60 cm



Material no.: 278698
Technical data:
 Width: 150 cm
 Height: 42 cm
 Depth: 60 cm

Accessories

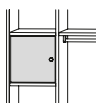


Wooden safe door
 Plastic safe box
 (without illustration)



Coat rack
 Material no.: 197373
Technical data:
 Width: 50 cm
 Height: 103 cm



Mirror
 Material no.: 197372
Technical data:
 Width: 50 cm
 Height: 103 cm

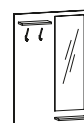

**Combination
 Mirror/coat rack**
 Material no.: 197374
Technical data:
 Width: 50 cm
 Height: 150 cm
 Depth: 22 cm


*If sideboards or cupboards are combined, the adjoining top surface of the sideboard ends flush with the sideboard body.

Decors




Natural Beech

R 24010 VV

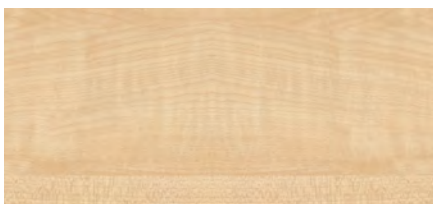


Royal Maple

R 27001 VV

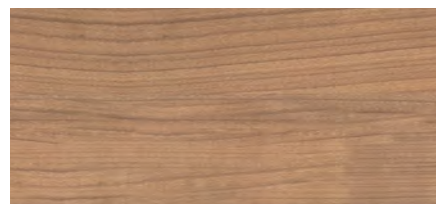


Havana Cherry

R 42006 VV

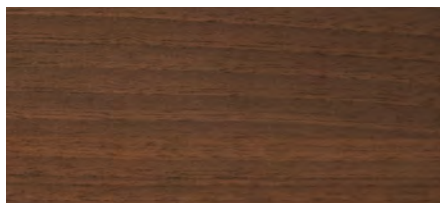

Bella Noce Choco

R 42006 VV

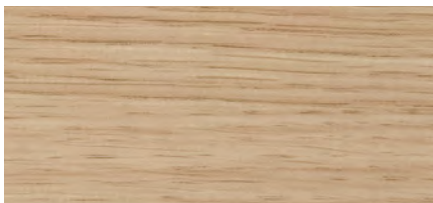

Lindberg Oak

R 20021 VV

Tables, coat rack, mirror (examples)



Pedestal table, round,
available in a range of diameters.



Dining table, rectangular,
available in various sizes.

Additional features for tables: Felt,
plastic or metal gliders.



Coat rack
(W: 50 cm, D: 103 cm).



Mirror
(W: 50 cm, D: 103 cm).

Delivery times
available on
request

Further information:



Product page

The Residenz chair range



Stacking chair

(W: 46 cm, H: 81 cm, D: 56 cm, seat height: 45 cm).



Stacking armchair

(W: 56 cm, H: 81 cm, D: 56 cm, seat height: 45 cm).



Mid-back armchair

(W: 60 cm, H: 91 cm, D: 58 cm, seat height: 45 cm).



High back armchair

(W: 60 cm, H: 104 cm, D: 56 cm, seat height: 45 cm).



Armchair

(W: 57 cm, H: 94 cm, D: 56 cm, seat height: 45 cm).



High back armchair

(W: 57 cm, H: 110 cm, D: 59 cm, seat height: 45 cm).

Covering fabric similar to illustration.

Additional equipment for seating:
Felt, plastic or metal gliders.

Minimum order
quantity: 10 chairs;
delivery times
available on request

Further information:



Product page



Custodians of values Shapers of the future

STIEGELMEYERGROUP

Custodians of values

Our company was founded in Herford in 1900 and is still a family-run enterprise today. Respect and a sense of responsibility towards our customers, employees and partners form the basis for our day-to-day work. Based on these values and our many years of experience, we offer our customers all over the world high-quality products, solutions and services to meet the needs of the care and healthcare sectors.

Shapers of the future

In order to cope with rapid changes in the care and healthcare sectors, we develop all our products ourselves. Creating innovations and visions of what care will be like in future are the goals we pursue. We maintain regular contact with our customers in order to support them in their daily tasks. Implementing tomorrow's trends today is a successful tradition of ours.

Our expertise

Hospital

Nursing home

Homecare

Lifestyle beds

Contract furniture

Service



The Stiegmeyer-Group is your international partner for hospitals and care facilities and, with the Burmeier brand, also for home care. We support our customers with practical products, solutions and services, which we constantly develop further.

- Founded in 1900 and still family-run today
- Employs 1,100 members of staff worldwide
- Production of all core products in our own factories
- Product development and manufacture in Germany
- Servicing, training and process optimisation at the customer's site
- International certification and export to more than 60 countries
- Sales companies in Belgium, Finland, France, the Netherlands, Poland and South Africa

Our brands





Burmeier GmbH & Co. KG

Industriestraße 53
32120 Hiddenhausen, Germany
Phone +49 (0) 5223 9769 - 0
Fax +49 (0) 5223 9769 - 090
info@burmeier.com
www.burmeier.com

A Stieglmeyer-Group company

STIEGELMEYERGROUP

Hauptsitz / Headquarters

Deutschland / Germany

Stieglmeyer GmbH & Co. KG

Ackerstraße 42
32051 Herford
Phone +49 (0) 5221 185 - 0
Fax +49 (0) 5221 185 - 252
info@stieglmeyer.com
www.stieglmeyer.com

France

Stieglmeyer SAS

16 Avenue de l'Europe
Bâtiment SXB1
67300 Schiltigheim
Phone +33 (0) 4 7238 - 5336
contact@stieglmeyer.fr
www.stieglmeyer.fr

South Africa

Stieglmeyer Africa (PTY) Ltd

The Estuaries · Century City
Cape Town, 7441
Phone +27 (0) 21 555 - 1415
Fax +27 (0) 21 555 - 0639
info.southafrica@stieglmeyer.com
www.stieglmeyer.co.za

Nederland / Netherlands

Stieglmeyer BV

Peppelenbos 7
6662 WB Elst
Phone +31 (0) 345 581 - 881
info@stieglmeyer.nl
www.stieglmeyer.nl

Suomi / Finland

Stieglmeyer Oy

Ahjokatu 4 A
15800 Lahti
Phone +358 (0) 3 7536 - 320
Fax +358 (0) 3 7536 - 330
stieglmeyer@stieglmeyer.fi
www.stieglmeyer.fi

Polska / Poland

Stieglmeyer Sp. z o.o.

86-212 Stolno
Grubno 63
Phone +48 56 6771 - 400
Fax +48 56 6771 - 410
info.stolno@stieglmeyer.pl
www.stieglmeyer.pl

